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1 introduction

1.1 Lens-manufacturer

the following manufacturers are able to receive data in this fileformat.

CZ = Carl Zeiss

ROD = Rodenstock

SEI = Seiko

ESS = Essilor

Chapter

2 XML-Struktur

2.1 ToDo-Liste

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2.2 overview

[Rootelement b2bOptic](#)

simple types

[AdditionRefractionMethod](#)

[AngleDimension](#)

[BevelPosType](#)

[BevelTypes](#)

[EdgingType](#)

[ChamferIntensity](#)

[ChamferPos](#)

[CoatingTypes](#)

[EdgingType](#)

[FrameMaterials](#)

[Gender](#)

[HeightReferences](#)

[MsgStates](#)

[MsgSteps](#)

[MsgTypes](#)

[OCReferences](#)

[Roles](#)

[Sides](#)

[SidesSimple](#)

[SoftwareTypes](#)

[ThicknessReferences](#)

[TracerBinaryFormat](#)

[TracerType](#)

complex types

[Account](#)

[Address](#)

[Bevel](#)

[Centration](#)

[Coating](#)

[ContactInfo](#)

[CPoint](#)

[Curve](#)

[Cylinder](#)

[Decentration](#)

[Diameter](#)

[DrillHoles](#)

[Edging](#)

[Frame](#)

[FrameSize](#)

[GeometryType](#)

[Header](#)

[IProfilerData](#)

[IProfilerMap](#)

[IProfilerResult](#)

[Item](#)

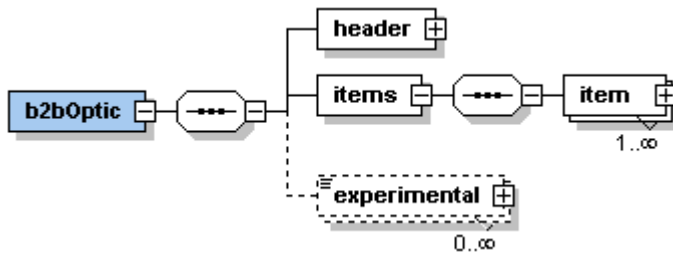
[Lens](#)

[Pair](#)

[Patient](#)

[PPoint](#)
[Prism](#)
[ProductCatalog](#)
[RXDataTypeSimple](#)
[RXDataType](#)
[RXDataType4SL](#)
[Shape](#)
[Software](#)
[StockLens](#)
[TimeStamps](#)
[TracerData](#)
[Zernikes](#)
[Zernikes7](#)
[Zernikes8](#)
[Zernikes9](#)
[Zernikes10](#)
[Zernikes11](#)
[Zernikes12](#)

2.3 rootelement "b2bOptic"

Elemente

header

type: [Header](#)
 occurs: 1
 description:

items

type: sequence of item
 occurs: 1
 description:

item

type: [Item](#)
 occurs: 1..n
 description:

experimental

type: anyType
 occurs: 0..n
 description:

XSD

```
<xs:element name="b2bOptic">
  <xs:complexType>
    <xs:sequence>
      <xs:element name="header" type="Header"/>
      <xs:element name="items">
        <xs:complexType>
          <xs:sequence>
            <xs:element name="item" type="Item" maxOccurs="unbounded"
              </xs:sequence>
          </xs:complexType>
        </xs:element>
        <xs:element name="experimental" type="xs:anyType" minOccurs="0" maxOccurs="unbounded"
          </xs:sequence>
        </xs:complexType>
      </xs:element>
    </xs:sequence>
  </xs:complexType>
</xs:element>
```

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Simple Type

type: string

possible values: MANUFACTURER
LAB
SHOP

description: MANUFACTURER:
LAB:
SHOP:

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possible values: CORNEA
FRAME

description:

info: CORNEA defines distance from eyerotationcenter to the cornea vertex
FRAME defines the distance from eyerotationcenter to the glass vertex (distance includes the HSA)

XSD

```
<xs:simpleType name="RCDReferences">
  <xs:restriction base="xs:string">
    <xs:enumeration value="CORNEA"/>
    <xs:enumeration value="FRAME"/>
  </xs:restriction>
</xs:simpleType>
```

2.4.22 Roles

Used in

[Header Item](#)

Simple Type

type: string

possible values: ORIGINATOR
SUPPLIER
SHIPTO
CARRIER
INVOICETO
PAIDBY
MANUFACTURER

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info: ROD: SUPPLIER,CARRIER, INVOICETO, PAIDBY, MANUFACTURER
are currently ignored
CZ: SUPPLIER,CARRIER, PAIDBY, MANUFACTURER are currently
ignored
CZ: SUPPLIER,CARRIER, INVOICETO, PAIDBY, MANUFACTURER
are currently ignored

XSD

```
<xs:simpleType name="Roles">
  <xs:restriction base="xs:string">
    <xs:enumeration value="ORIGINATOR" />
    <xs:enumeration value="SUPPLIER" />
    <xs:enumeration value="SHIPTO" />
    <xs:enumeration value="CARRIER" />
    <xs:enumeration value="INVOICETO" />
    <xs:enumeration value="PAIDBY" />
    <xs:enumeration value="MANUFACTURER" />
  </xs:restriction>
</xs:simpleType>
```

2.4.23 RxDataReferences

Used in

[RxDataTypeFarNear](#)

Simple Type

type: string

possible values: OBJECTIVE
SUBJECTIVE

description:

XSD

```
<xs:simpleType name="RxDataReferences">
  <xs:restriction base="xs:string">
    <xs:enumeration value="OBJECTIVE"/>
    <xs:enumeration value="SUBJECTIVE"/>
  </xs:restriction>
</xs:simpleType>
```

2.4.24 Sides

Used in

[Bevel Frame](#) [IProfilerResult](#) [Item](#) [Lens](#)

Simple Type

type: string

possible values: RIGHT
LEFT
UNDEFINED

description:

info: SEI: UNDEFINED not used

XSD

```
<xs:simpleType name="Sides">  
  <xs:restriction base="xs:string">  
    <xs:enumeration value="RIGHT"/>  
    <xs:enumeration value="LEFT"/>  
    <xs:enumeration value="UNDEFINED"/>  
  </xs:restriction>  
</xs:simpleType>
```

2.4.25 SidesSimple

Used in
[Frame](#)

Simple Type

type: string

possible values: RIGHT
LEFT

description:

info:

XSD

```
<xs:simpleType name="Sides">  
  <xs:restriction base="xs:string">  
    <xs:enumeration value="RIGHT"/>  
    <xs:enumeration value="LEFT"/>  
  </xs:restriction>  
</xs:simpleType>
```

2.4.26 SoftwareTypes

Used in

[Software](#)

Simple Type

type: string

possible values: ORIGINATOR
VERIFIER
SENDER

description: ORIGINATOR: LngDe: Erzeuger der XML
VERIFIER: LngDe: Prüfsoftware
SENDER: LngDe: Sender der XML

XSD

```
<xs:simpleType name="SoftwareTypes">  
  <xs:restriction base="xs:string">  
    <xs:enumeration value="ORIGINATOR" />  
    <xs:enumeration value="VERIFIER" />  
    <xs:enumeration value="SENDER" />  
  </xs:restriction>  
</xs:simpleType>
```

2.4.27 ThicknessReferences

Used in

[GeometryType](#)

Simple Type

type: string

possible values: CENTER
EDGE
DRILLHOLE

description:

CENTER = Mindestmittendicke
EDGE = Mindestranddicke
DRILLHOLE = Mindestdicke am Bohrloch

XSD

```
<xs:simpleType name="ThicknessReferences">  
  <xs:restriction base="xs:string">  
    <xs:enumeration value="CENTER"/>  
    <xs:enumeration value="EDGE"/>  
    <xs:enumeration value="DRILLHOLE"/>  
  </xs:restriction>  
</xs:simpleType>
```

2.4.28 ThicknessReductionReferences

Used in

[GeometryType](#)

Simple Type

type: string

possible values: REDUCEWITHSHAPE
REDUCEWITHSHAPETOROUND

description:

XSD

2.4.29 TracerBinaryFormat

Used in
[TracerData](#)

Simple Type

type: string

possible values: DLL Briot
 DVI
 GT3000
 MO1
 NIDEK
 OMA3.02
 PHI
 T4
 WECO

description:

additional info:

Brand	Model	Description	Interface	AdditionalInfos
BRIOT	SCANNET	Scannet 2 Shapes Reader	OMA 3,02	512 radius, with z values
ESSILOR	PHI	Essilor Phi Shapes Reader	PHI	400 radius, with 200 z values
		Essilor Phi Shapes Reader	OMA 3,02	512 radius, with z values
	KAPPA	Essilor Kappa Tracer and Blocker	PHI	400 radius, with 200 z values
		Essilor Kappa Tracer and Blocker	OMA 3,02	512 radius, with 512 z values
WECO	Trace 1/2	Old Weco Tracer	DVI?	512 Radius
		Old Weco Tracer	Nidek	1000 Radius
HOYA	GT1000	Hoya GT1000	GT3000	500 radius
			OMA 3,02	512 radius, z values has to be clarified
HOYA	GT3000	Hoya GT3000	GT3000	500 radius
			OMA 3,02	512 radius, z values has to be clarified
NIDEK	LT 900SX	LT 900 SX Shapes Reader	Nidek	1000 Radius
SCHOENE	INDO	Schoene INDO Shapes Reader	Nidek	1000 Radius

Priority	Brand	Model	Description	Interface	Information	Implemented
3	BRIOT	SCANFROM	Scanform 1 Shapes Reader	DLL BRIOT	1024 radius	no
1	BRIOT	AXCEL	Blocker with Tracer	OMA 3,02	512 radius, with 512 z values	no
3	OPTRONIC	T4	Optronics T4 Shapes Reader	T4	400 radius, with 400 z values	no
1	WECO	TRACER 3	Weco Tracer 3 Shapes Reader	OMA 3,02	512 radius, with 512 z values	no
1	WECO	CAD 5	Blocker with Tracer from WECO	OMA 3,02	512 radius, with 512 z values	no
1	NIDEK	SX 9000	SX 9000 Edger	DVI	500 Radius	no
3	NIDEK	SX 7070	SX 7070 Edger	DVI	500 Radius	no
1	NIDEK	LT 900 SX	LT 900 SX Shapes Reader	DVI	500 Radius	no
3	NIDEK	LT 700 SX	LT 700 SX Shapes Reader	DVI	500 Radius	no
1	NIDEK	LT 900 SX VCA	LT 900 SX VCA Shapes Reader	OMA 3,02	1000 Radius	no
1	R.O.M.	PIETRO	FBK	NIDEK	1000 Radius	no

XSD

```

<xs:simpleType name="TracerBinaryFormat">
  <xs:restriction base="xs:string">
    <xs:enumeration value="DLL BRIOT" />
    <xs:enumeration value="DVI" />
    <xs:enumeration value="GT3000" />
    <xs:enumeration value="MO1" />
    <xs:enumeration value="NIDEK" />
    <xs:enumeration value="OMA3.02" />
    <xs:enumeration value="PHI" />
    <xs:enumeration value="T4" />
    <xs:enumeration value="WECO" />
  </xs:restriction>
</xs:simpleType>

```

2.4.30 TracerType

Used in
[TracerData](#)

Simple Type

type: string

possible values:

- Hoya 3DFT
- Hoya GT1000
- Hoya GT3000
- Hoya GT3000 OMA
- Hoya UT1000
- Essilor Gamma
- Essilor PHI PCCOM
- Essilor PHI OMA
- Essilor Kappa PCCOM
- Essilor Kappa OMA
- National Optronics 4T
- Briot ScanfomNet I
- Briot ScanfomNet II
- Weco 3DFT+
- Weco Trace II
- Weco Trace III
- Nidek LT700
- Nidek LT900
- Nidek LT900 OMA
- Nidek LT1000
- Indo NDK
- Indo OMA
- Topcon FR50
- Takubomatic FD80
- Unknown

description:

XSD

```
<xs:simpleType name="TracerType">
  <xs:restriction base="xs:string">
    <xs:enumeration value="Hoya 3DFT" />
    <xs:enumeration value="Hoya GT1000" />
    <xs:enumeration value="Hoya GT3000" />
    <xs:enumeration value="Hoya GT3000 OMA" />
    <xs:enumeration value="Hoya UT1000" />
    <xs:enumeration value="Essilor Gamma" />
    <xs:enumeration value="Essilor PHI PCCOM" />
    <xs:enumeration value="Essilor PHI OMA" />
    <xs:enumeration value="Essilor Kappa PCCOM" />
    <xs:enumeration value="Essilor Kappa OMA" />
    <xs:enumeration value="National Optronics 4T" />
  </xs:restriction>
</xs:simpleType>
```

```
<xs:enumeration value="Briot ScanfomNet I" />
<xs:enumeration value="Briot ScanfomNet II" />
<xs:enumeration value="Weco 3DFT+" />
<xs:enumeration value="Weco Trace II" />
<xs:enumeration value="Weco Trace III" />
<xs:enumeration value="Nidek LT700" />
<xs:enumeration value="Nidek LT900" />
<xs:enumeration value="Nidek LT900 OMA" />
<xs:enumeration value="Nidek LT1000" />
<xs:enumeration value="Indo NDK" />
<xs:enumeration value="Indo OMA" />
<xs:enumeration value="Topcon FR50" />
<xs:enumeration value="Takubomatic FD80" />
<xs:enumeration value="Unknown" />
</xs:restriction>
</xs:simpleType>
```

2.5 Complex Types

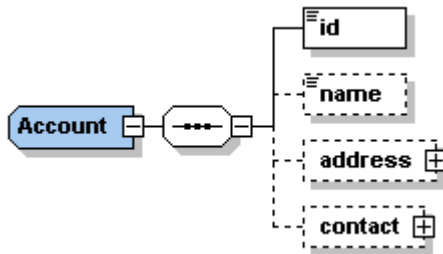
complex types

[Account](#)
[Address](#)
[Bevel](#)
[Centration](#)
[Coating](#)
[ContactInfo](#)
[CPoint](#)
[Curve](#)
[Cylinder](#)
[Cylinder](#)
[Decentration](#)
[Diameter](#)
[DrillHoles](#)
[Edging](#)
[Frame](#)
[FrameSize](#)
[GeometryType](#)
[Header](#)
[IProfilerData](#)
[IProfilerMap](#)
[IProfilerResult](#)
[Item](#)
[Lens](#)
[Pair](#)
[Patient](#)
[PPoint](#)
[Prism](#)
[ProductCatalog](#)
[RXDataType](#)
[RXDataType4SL](#)
[Shape](#)
[Software](#)
[StockLens](#)
[TimeStamps](#)
[TracerData](#)
[Zernikes](#)
[Zernikes7](#)
[Zernikes8](#)
[Zernikes9](#)
[Zernikes10](#)
[Zernikes11](#)
[Zernikes12](#)

2.5.1 Account

Used in

[Header Item Pair](#)

Elemente

id

type: string

attribute: memberShipID (Type long, default=1)

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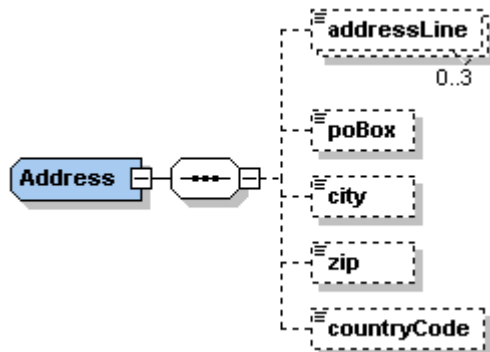
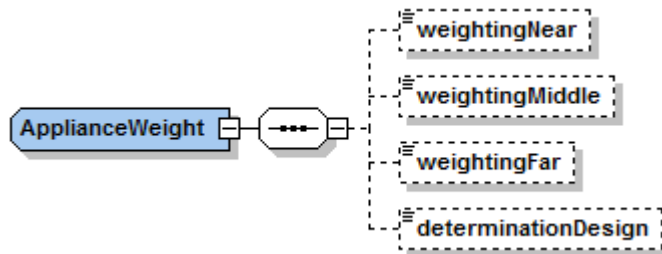
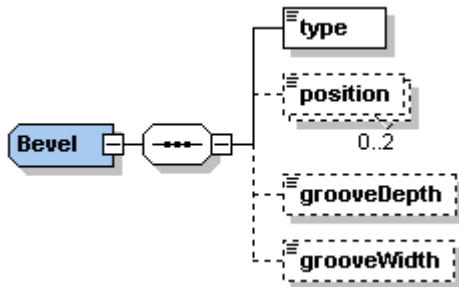
Elemente

type

type: [BevelTypes](#)
occurs: 1
description:

position

type: float
unity: mm / % / dpt
attribute: posType ([BevelPosType](#))
attribute: side ([Sides](#))
occurs: 0..2
description:
info: SEI: FRONT/BACK: Werte von 0,1mm bis 2,00 mm
 SEI: RELATED: Werte von 30% bis 70%
 CZ: attribute FRAMECURVATURE and FRAMEBASE is not supported
 CZ: In case of FRONT, BACK: [mm] distance to front/back.
 CZ: In case of RELATED: [%] front/back-distance

grooveDepth

type: float
unity: mm
occurs: 0..1
description:
info: CZ: ignored

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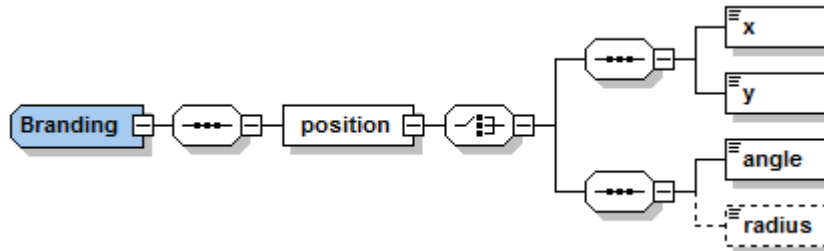
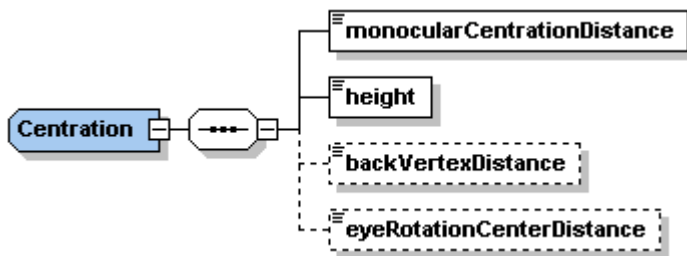
info: CZ: Attribut reference: Currently only FAR supported, NEAR will be ignored!

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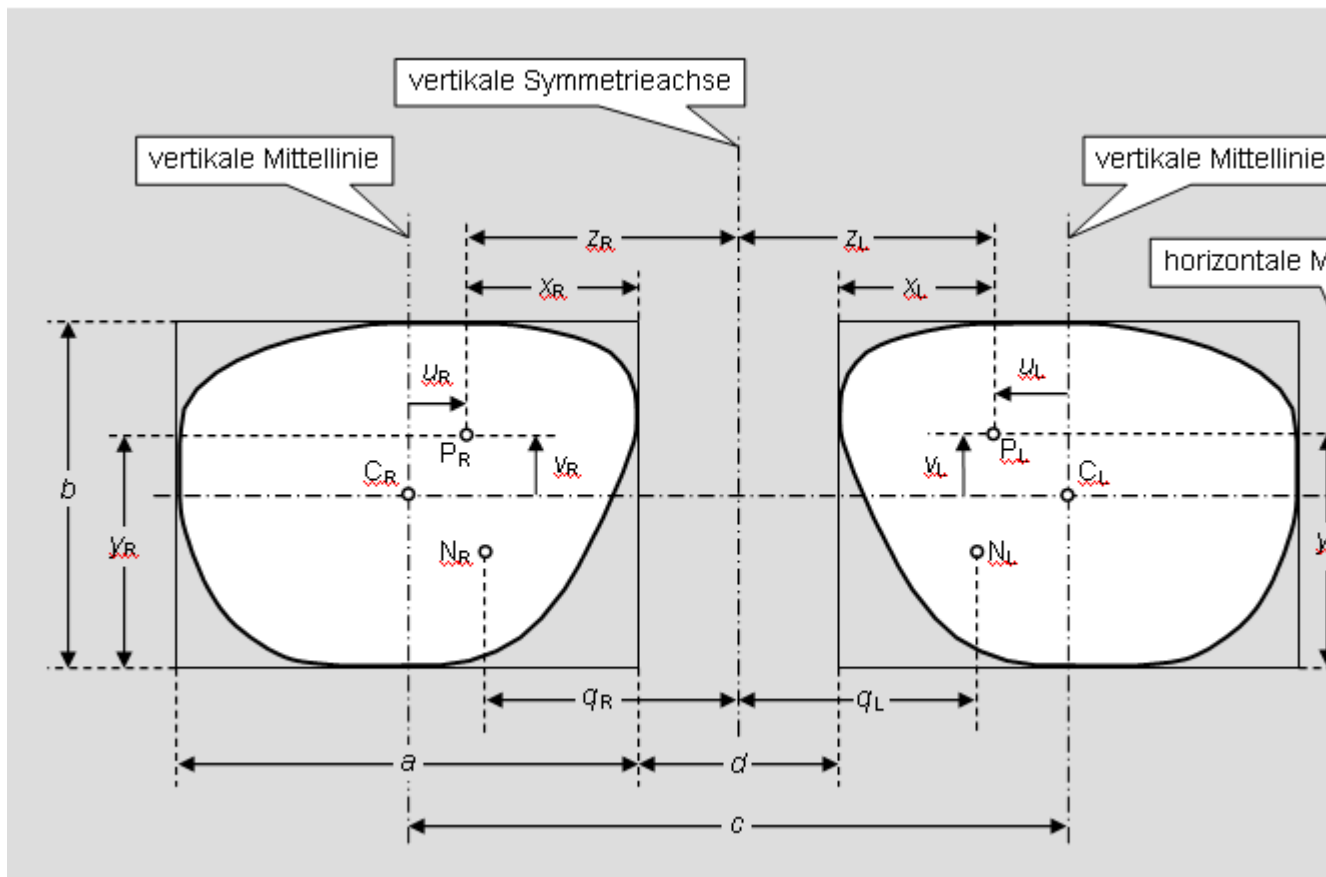
info: CZ: Attribut reference: FAR -> Y Near -> H

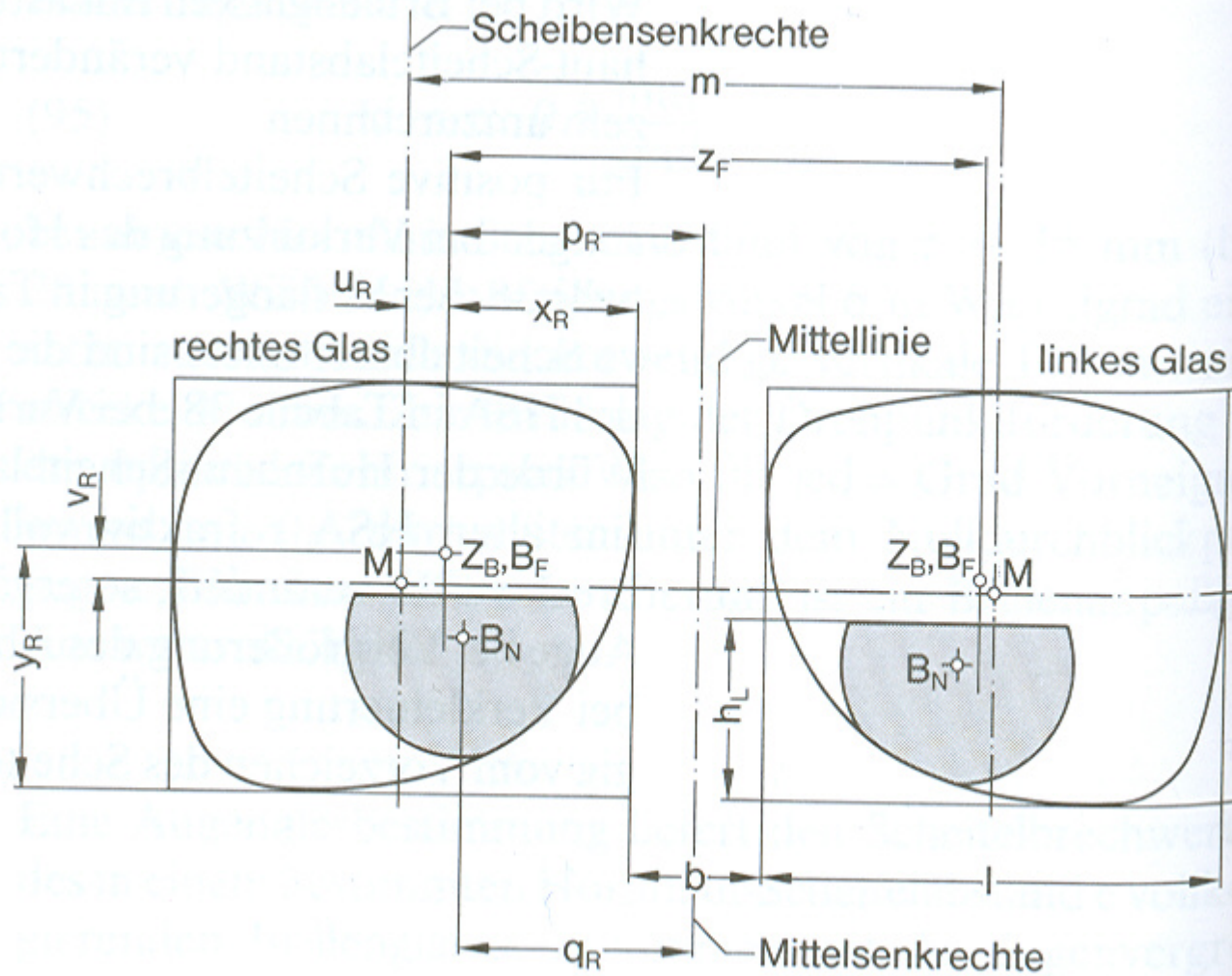
backVertexDistance

type: float
unity: mm
attribute: rxDataNeedRecalculation:boolean
occurs: 0..1
description: Hornhautscheitelabstand

eyeRotationCenterDistance

type: float
unity: mm
attribute: reference ([RCDReferences](#)) - required
occurs: 0..1
description: "Abstand Augendrehpunkt zum Referenzpunkt (Attribut)"
info: CZ: Supports only attribute "CORNEA"

XSD

```

<xs:complexType name="Centration">
  <xs:sequence>
    <xs:element name="monocularCentrationDistance">
      <xs:complexType>
        <xs:simpleContent>
          <xs:extension base="xs:float">
            <xs:attribute name="reference" type="
  </xs:extension>
  </xs:simpleContent>
  </xs:complexType>
</xs:element>
<xs:element name="height">
  <xs:complexType>
    <xs:simpleContent>
      <xs:extension base="xs:float">

```

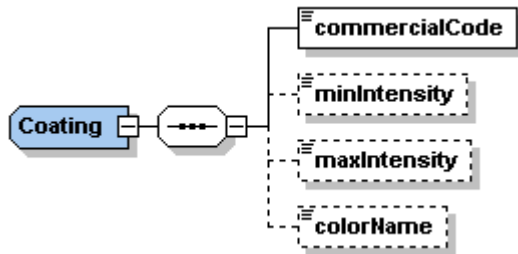
```
OCReferences" use="required" />
HeightReferences" use="required" />
</xs:extension>
</xs:simpleContent>
</xs:complexType>
</xs:element>
<xs:element name="backVertexDistance" type="xs:float" minOccurs="0" />
<xs:element name="eyeRotationCenterDistance" minOccurs="0">
  <xs:complexType>
    <xs:simpleContent>
      <xs:extension base="xs:float">
        <xs:attribute name="reference" type="
RCDReferences" use="required" />
</xs:extension>
</xs:simpleContent>
</xs:complexType>
</xs:element>
</xs:sequence>
</xs:complexType>
```

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unity: percentage

occurs: 0..1

description: Minimum color intensity for tinting as a percentage

info: CZ: ignored

maxIntensity

type: integer

unity: percentage

occurs: 0..1

description: Maximum color intensity for tinting as a percentage

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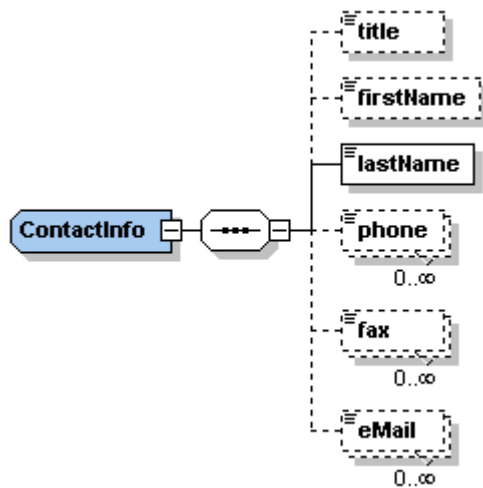
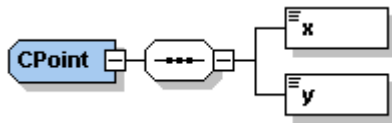
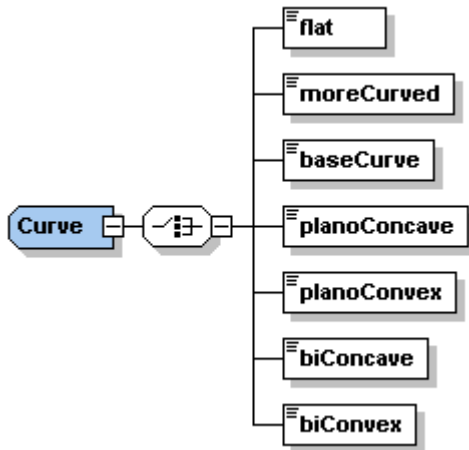
baseCurve

type: float
unity: dpt
occurs: 1
description: LngDE: Basiskurve
info: CZ: ignored

planoConcave

type: boolean
occurs: 1
description:
info: CZ: ignored

planoConvex

type: boolean
occurs: 1
description:
info: CZ: ignored

biConcave

type: boolean
occurs: 1
description:
info: CZ: ignored

biConvex

type: boolean
occurs: 1
description:
info: CZ: ignored

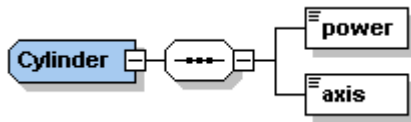
XSD

```
<xs:complexType name="Curve">
  <xs:choice>
    <xs:element name="flat" type="xs:boolean"/>
    <xs:element name="moreCurved" type="xs:boolean"/>
    <xs:element name="baseCurve" type="xs:float"/>
    <xs:element name="planoConcave" type="xs:boolean"/>
    <xs:element name="planoConvex" type="xs:boolean"/>
    <xs:element name="biConcave" type="xs:boolean"/>
    <xs:element name="biConvex" type="xs:boolean"/>
  </xs:choice>
</xs:complexType>
```

2.5.11 Cylinder

Used in

[RefractionSimple](#) [RXDataType](#) [RXDataType4SL](#)

Elemente

power

type: float
unity: dpt
occurs: 1
description:

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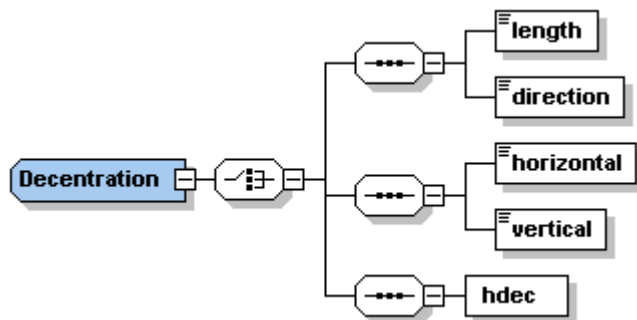
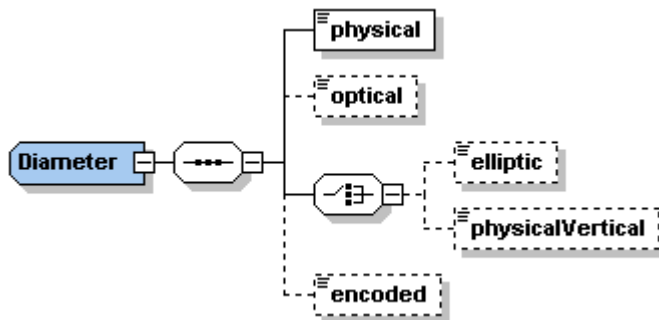
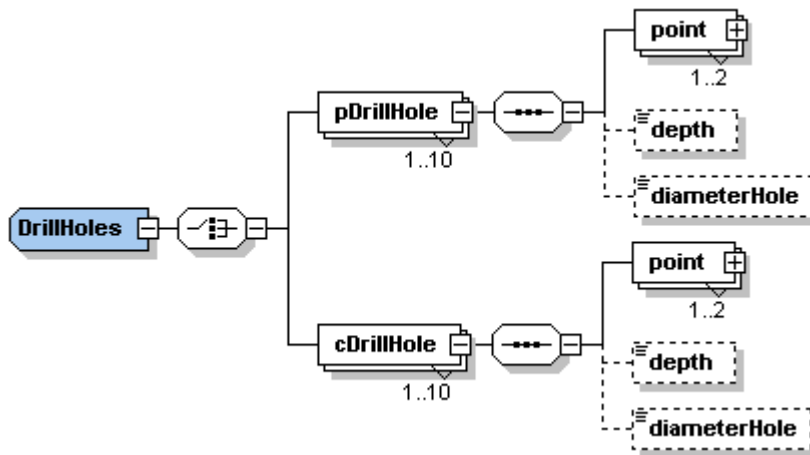
Elemente

pDrillHole

type: complex
occurs: 1..10
description:
info: CZ: ignored - use cDrillHole instead

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depth

type: float
unity: mm
occurs: 0..1
description:

diameterHole

type: float
unity: mm
occurs: 0..1
description:

cDrillHole

type: complex
occurs: 1..10
description:
info: CZ: supports only 4 drillholes (longholes)

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depth

type: float
unity: mm
occurs: 0..1
description:
info: CZ: ignored

diameterHole

type: float
unity: mm
occurs: 0..1
description:
info: CZ: ignored

XSD

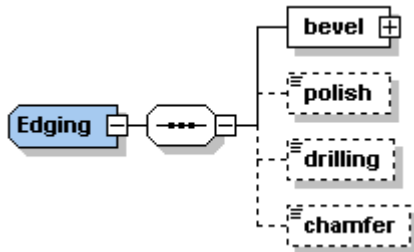
```

<xs:complexType name="DrillHoles">
  <xs:choice>
    <xs:element name="pDrillHole" maxOccurs="10">
      <xs:complexType>
        <xs:sequence>
          <xs:element name="point" type="PPoint" maxOccurs="2"/>
          <xs:element name="depth" type="xs:float" minOccurs="0"
minOccurs="0"/>
          <xs:element name="diameterHole" type="xs:float"
        </xs:sequence>
      </xs:complexType>
    </xs:element>
    <xs:element name="cDrillHole" maxOccurs="10">
      <xs:complexType>
        <xs:sequence>
          <xs:element name="point" type="CPoint" maxOccurs="2"
minOccurs="0"/>
          <xs:element name="depth" type="xs:float" minOccurs="0"
minOccurs="0"/>
          <xs:element name="diameterHole" type="xs:float"
        </xs:sequence>
      </xs:complexType>
    </xs:element>
  </xs:choice>
</xs:complexType>

```

```
</xs:choice>  
</xs:complexType>
```

2.5.15 Edging

Used in

[Pair](#)

Attribute

edgingType

type: [EdgingType](#)

use: required

description:

info:

reTrace

type: boolean

use: required

description:

info:

Elemente

bevel

type: [Bevel](#)

occurs: 1

description:

drilling

type: boolean

occurs: 0..1

description: in value=true then "/Frame/drillHoles" is used for drilling. Otherwise "/Frame/drillHoles" is just for information.

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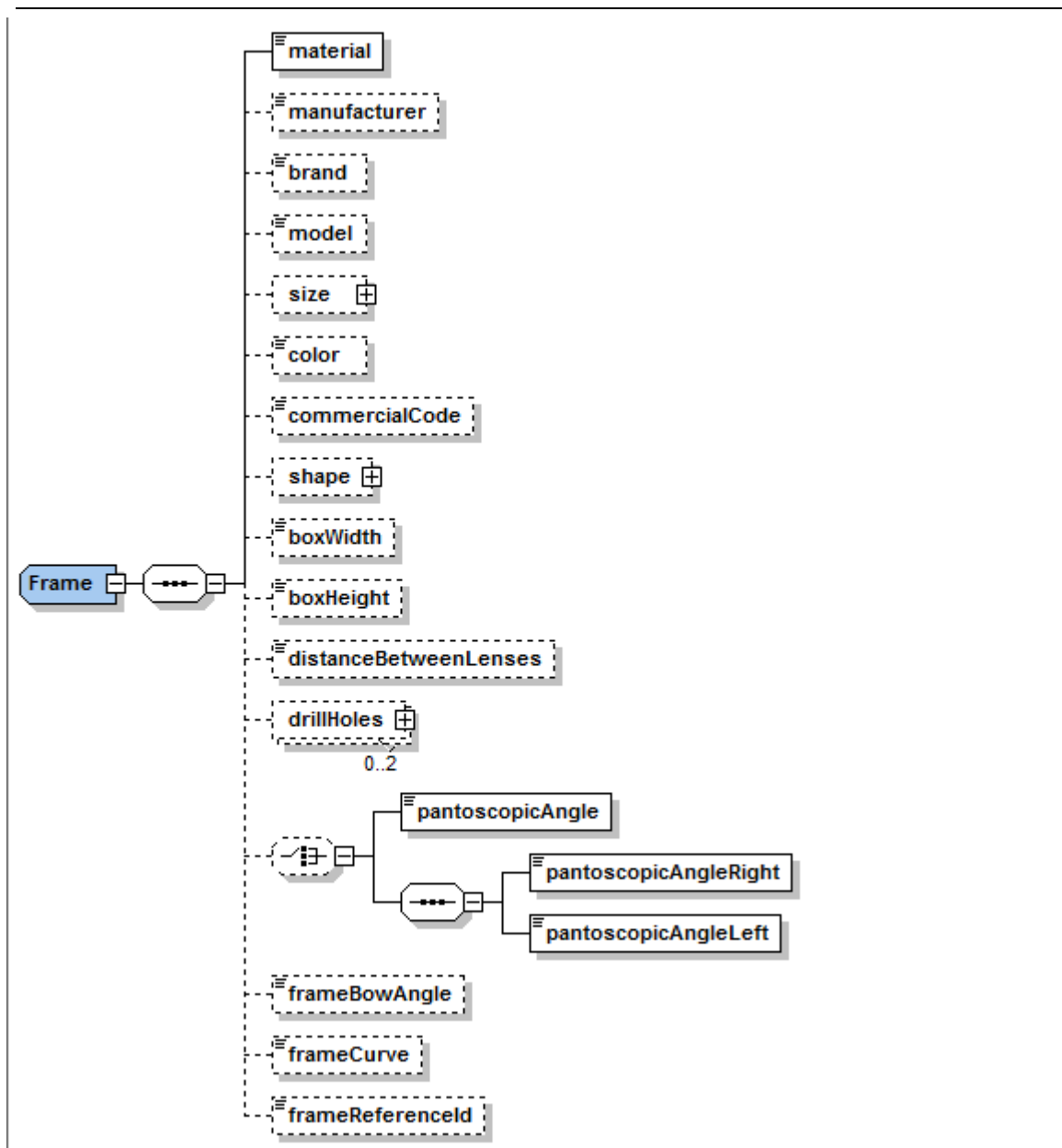
occurs: 1

description:

info: CZ: currently recognized: METAL, PLASTIC, NYLOR, SPECIAL

manufacturer

type: string

occurs: 0..1

description:

brand

type: string

occurs: 0..1

description:

info: CZ: ignored

model

type: string

occurs: 0..1

description:

size

type: [FrameSize](#)

occurs: 0..1

description: Nominal size information as given from frame manufacturer

info: CZ: currently ignored - use size elements under shape!

color

type: string

occurs: 0..1

description:

info: CZ: ignored

commercialCode

type: string

occurs: 0..1

description:

info: CZ: ignored

shape

type: complex

occurs: 0..1

description:

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catalog

type:

occurs: 1..2

description: for thicknessreduction

Info: CZ: ignored

Elemente für catalog

shapeld

type: string
occurs: 1
description:

explicit

type: [Shape](#)
attribute: side ([SidesSimple](#))
occurs: 1..2
description:

traceData

type: [TracerData](#)
occurs: 1..2
description: necessary for remoteEdging/Presize

boxWidth

type: float
unity: mm
occurs: 0..1
description: measured box width for size transformations

boxHeight

type: float
unity: mm
occurs: 0..1
description: measured box height for size transformations

DistanceBetweenLenses

type: float
unity: mm
occurs: 0..1
description: measured dbl for calculation purposes

drillHoles

type: [DrillHoles](#)
attribute: side ([Sides](#))
occurs: 0..2
description:
info: CZ: If attribute side is UNDEFINED and drillHoles occurs only once the drillhole values are mirrored from right to left side

SEI: ignored

pantoscopicAngle

type: float
attribute: dimension ([AngleDimension](#))
occurs: 0..1
description: LngDE: Fassungsverneigung
info: ROD:possible value for AngleDimension: MM and DEG

pantoscopicAngleRight

type: float
attribute: dimension ([AngleDimension](#))
occurs: 0..1
description: LngDE: Fassungsvoeneigung (rechte Seite)
info: ROD:possible value for AngleDimension: MM and DEG

pantoscopicAngleLeft

type: float
attribute: dimension ([AngleDimension](#))
occurs: 0..1
description: LngDE: Fassungsvoeneigung (linke Seite Seite)
info: ROD:possible value for AngleDimension: MM and DEG

frameBowAngle

type: float
attribute: dimension ([AngleDimension](#))
occurs: 0..1
description: LangDe: Fassungs-scheibenwinkel
info:

frameCurve

type: float
unity: dpt
occurs: 0..1
description:
info: CZ: ignored

frameReferenceId

type: string
occurs: 0..1
description: todo:Beschreibung
info: CZ: ignored

XSD

```

<xs:complexType name="Frame">
  <xs:sequence>
    <xs:element name="material" type="FrameMaterials"/>
    <xs:element name="manufacturer" type="xs:string" minOccurs="0"/>
    <xs:element name="brand" type="xs:string" minOccurs="0"/>
    <xs:element name="model" type="xs:string" minOccurs="0"/>
    <xs:element name="size" type="FrameSize" minOccurs="0"/>
    <xs:element name="color" type="xs:string" minOccurs="0"/>
    <xs:element name="commercialCode" type="xs:string" minOccurs="0"/>
    <xs:element name="shape" minOccurs="0">
      <xs:complexType>
        <xs:sequence>
          <xs:choice>
            <xs:element name="catalog">
              <xs:complexType>
                <xs:sequence>
  
```

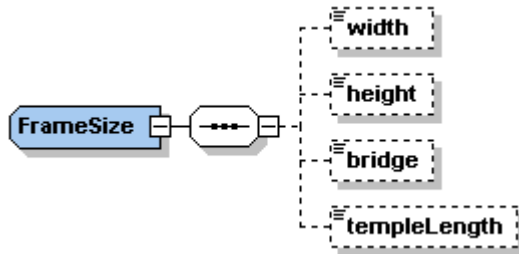
```

shaped" type="xs:string" />
</xs:sequence>
</xs:complexType>
</xs:element>
<xs:element name="explicit" maxOccurs="2">
  <xs:complexType>
    <xs:complexContent>
      <xs:extension base="
Shape">
      </xs:extension>
    </xs:complexContent>
  </xs:complexType>
  </xs:element>
  <xs:element name="tracerData" type="TracerData
"/>
  </xs:choice>
  </xs:sequence>
</xs:complexType>
</xs:element>
<xs:element name="boxWidth" type="xs:float" minOccurs="0" />
<xs:element name="boxHeight" type="xs:float" minOccurs="0" />
<xs:element name="distanceBetweenLenses" type="xs:float" minOccurs="0" />
<xs:element name="drillHoles" minOccurs="0" maxOccurs="2">
  <xs:complexType>
    <xs:complexContent>
      <xs:extension base="DrillHoles">
        <xs:attribute name="side" type="Sides" use="
required" />
      </xs:extension>
    </xs:complexContent>
  </xs:complexType>
</xs:element>
<xs:element name="pantoscopicAngle" minOccurs="0">
  <xs:complexType>
    <xs:simpleContent>
      <xs:extension base="xs:float">
        <xs:attribute name="dimension" type="
AngleDimension" use="optional" default="DEG" />
      </xs:extension>
    </xs:simpleContent>
  </xs:complexType>
</xs:element>
<xs:element name="frameBowAngle" minOccurs="0">
  <xs:complexType>
    <xs:simpleContent>
      <xs:extension base="xs:float">
        <xs:attribute name="dimension" type="
AngleDimension" use="optional" default="DEG" />
      </xs:extension>
    </xs:simpleContent>
  </xs:complexType>
</xs:element>
<xs:element name="frameCurve" type="xs:float" minOccurs="0" />
</xs:sequence>
<xs:attribute name="quantity" type="xs:integer" use="required" />
</xs:complexType>

```

2.5.18 FrameSize

Used in
[Frame](#)

Elemente

width

type: float
unity: mm
occurs: 0..1
description:

height

type: float
unity: mm
occurs: 0..1
description:

bridge

type: float
unity: mm
occurs: 0..1
description:

templeLength

type: float
unity: mm
occurs: 0..1
description:

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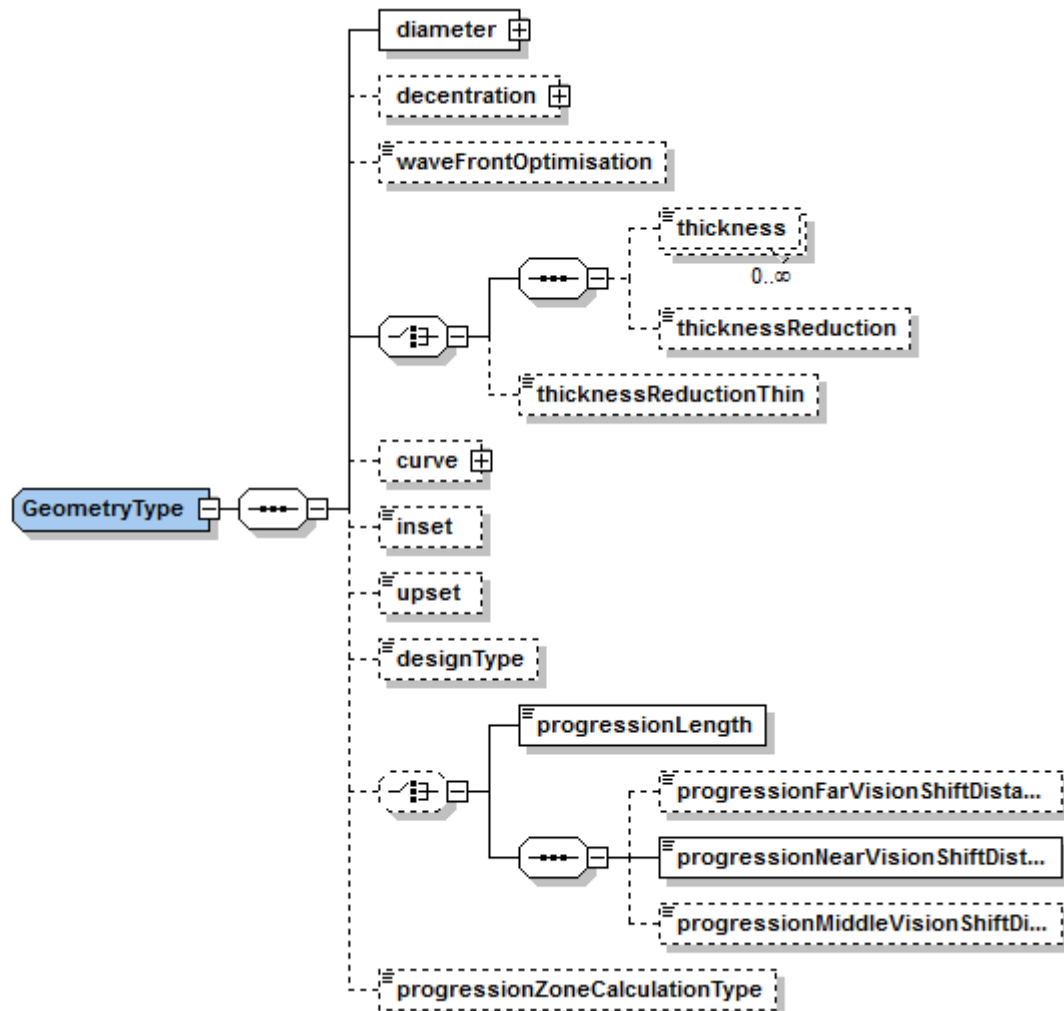
description:

thickness

type: float
unity: mm
attribute: reference ([ThicknessReferences](#))
occurs: 0..1
description:
info: CZ: attribute REDUCEWITHSHAPE is not supported
this is mapped with "thicknessReduction"

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thicknessReductionThin

type: boolean
occurs: 0..1
description: LngDe: Randscharf
info:

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unity: mm

occurs: 0..1

description: for progressive lenses with variable length of progressionzone

info: CZ: ignored. using "frameFit" under "lens"

progressionFarVisionShiftDistance

type: float

unity: mm

occurs: 0..1

description: Far Vision Vertical Shift distance Bz to Bf

info: CZ: ignored

SEI: ignored

progressionNearVisionShiftDistance

type: float

unity: mm

occurs: 1

description: Near Vision Vertical Shift distance Bz to Bn

info: CZ: ignored

SEI: ignored

progressionMiddleVisionShiftDistance

type: float

unity: mm

occurs: 0..1

description: todo

info: CZ: ignored

SEI: ignored

progressionZoneCalculationType

type: string

occurs: 0..1

description: Type of calculation of the progression zone length

info: CZ: ignored

SEI: ignored

XSD

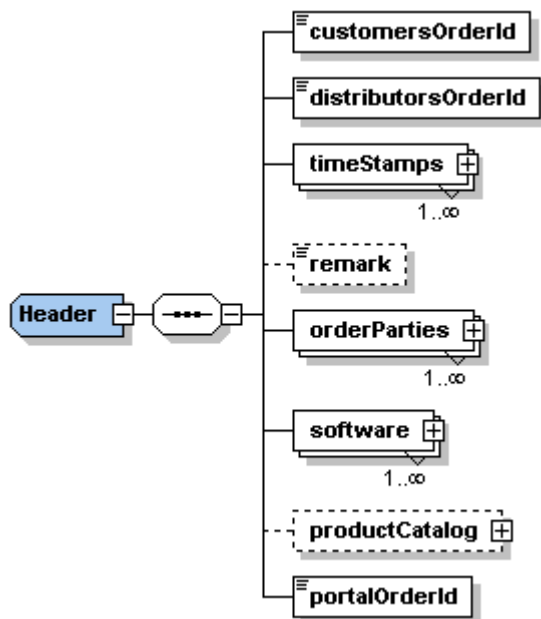
```

<xs:complexType name="GeometryType">
  <xs:sequence>
    <xs:element name="diameter" type="Diameter" />
    <xs:element name="decentration" type="Decentration" minOccurs="0" />
    <xs:element name="waveFrontOptimisation" type="xs:boolean" default="false"
minOccurs="0" />
    <xs:choice>
      <xs:sequence>
        <xs:element name="thickness" minOccurs="0" maxOccurs="
unbounded">
          <xs:complexType>
            <xs:simpleContent>
              <xs:extension base="xs:float">
                <xs:attribute name="reference"
type="ThicknessReferences" use="required" />
              </xs:extension>
            </xs:simpleContent>
          </xs:complexType>
        </xs:element>
        <xs:element name="thicknessReduction" type="xs:boolean" default
="false" minOccurs="0" />
      </xs:sequence>
      <xs:element name="thicknessReductionThin" type="xs:boolean" default="
false" minOccurs="0" />
    </xs:choice>
    <xs:element name="curve" type="Curve" minOccurs="0" />
    <xs:element name="inset" type="xs:float" minOccurs="0" />
    <xs:element name="upset" type="xs:float" minOccurs="0" />
    <xs:element name="designType" minOccurs="0">
      <xs:simpleType>
        <xs:restriction base="xs:string">
          <xs:enumeration value="A" />
          <xs:enumeration value="B" />
          <xs:enumeration value="C" />
        </xs:restriction>
      </xs:simpleType>
    </xs:element>
    <xs:choice minOccurs="0">
      <xs:element name="progressionLength" type="xs:float" />
      <xs:sequence>
        <xs:element name="progressionFarVisionShiftDistance" type="
xs:float" />
        <xs:element name="progressionNearVisionShiftDistance" type="
xs:float" />
        <xs:element name="progressionMiddleVisionShiftDistance" type="
xs:float" minOccurs="0" />
      </xs:sequence>
    </xs:choice>
    <xs:element name="progressionZoneCalculationType" type="xs:string" minOccurs="0"
"/>
  </xs:sequence>
</xs:complexType>

```

2.5.20 Header

Used in

[b2bOptic](#)

Attribute

msgType

type: [MsgTypes](#)
 use: optional
 default: REQUEST
 description:

msgState

type: [MsgStates](#)
 use: optional
 default: NEW
 description:

testIndicator

type: boolean
 use: optional
 default: 0
 description:

Elemente

customersOrderId

type: string
occurs: 1
description:
info: CZ: ignored

distributorsOrderId

type: string
occurs: 1
description:
info: CZ: ignored

timeStamps

type: [TimeStamps](#)
occurs: 1..n
description:
info: CZ: ignored

remark

type: string
occurs: 1
description:
info: CZ: Any comments or remarks are effectively ignored!

orderParties

type: [Account](#)
attribute: role (Type [Roles](#))
use: yes
occurs: 1..n
description: (LngDe) orderParties ist eine Ableitung von Account mit einem zusätzlichen Attribut role vom Type Roles.
info: CZ: at least one entry with @role ORIGINATOR or INVOICETO should be present, unless present under items/parties; serves as default - if nothing is specified in items branch, these entries are used

software

type: [Software](#)
occurs: 1..n
description:

productCatalog

type: [ProductCatalog](#)
occurs: 1
description:
info: SEI: ignored

portalOrderId

type: string
occurs: 1
description: retailer's order id
(LngDe)Bestellnummer des Optikers)
info: CZ: ignored

XSD

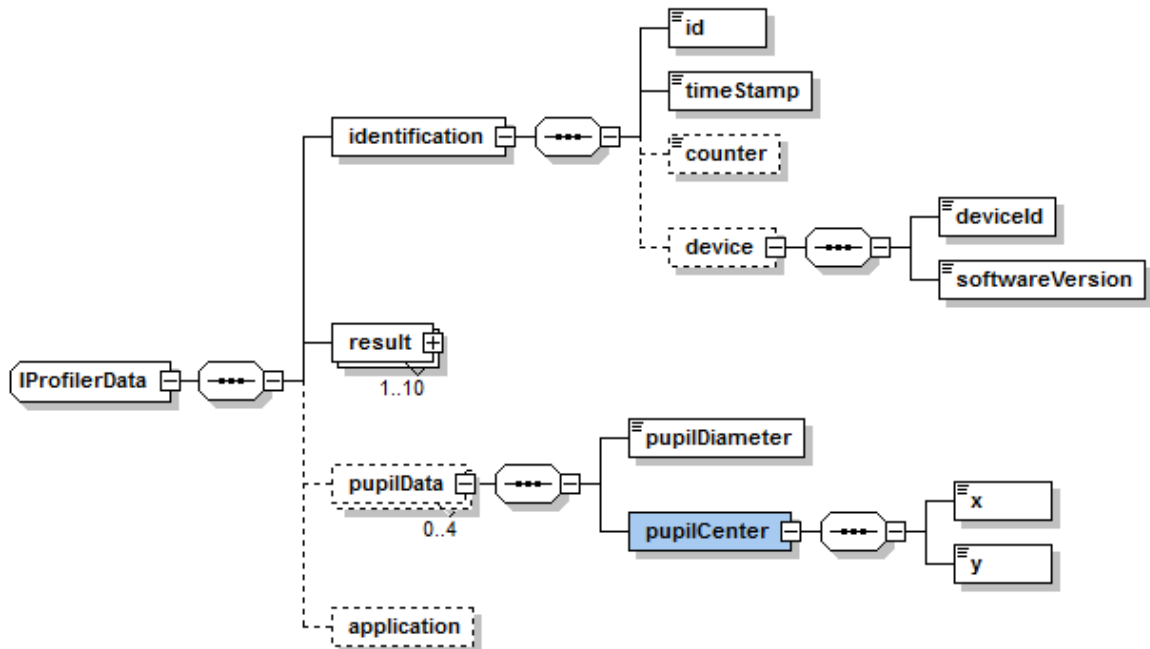
```

<xs:complexType name="Header">
  <xs:sequence>
    <xs:element name="customersOrderId" type="xs:string"/>
    <xs:element name="distributorsOrderId" type="xs:string"/>
    <xs:element name="timeStamps" type="TimeStamps" maxOccurs="unbounded"/>
    <xs:element name="remark" type="xs:string" minOccurs="0"/>
    <xs:element name="orderParties" maxOccurs="unbounded">
      <xs:complexType>
        <xs:complexContent>
          <xs:extension base="Account">
            <xs:attribute name="role" type="Roles" use="
required"/>
          </xs:extension>
        </xs:complexContent>
      </xs:complexType>
    </xs:element>
    <xs:element name="software" type="Software" maxOccurs="unbounded"/>
    <xs:element name="productCatalog" type="ProductCatalog" minOccurs="0"/>
    <xs:element name="portalOrderId" type="xs:string"/>
  </xs:sequence>
  <xs:attribute name="msgType" type="MsgTypes" use="optional" default="REQUEST"/>
  <xs:attribute name="msgState" type="MsgStates" use="optional" default="NEW"/>
  <xs:attribute name="testIndicator" type="xs:boolean" use="optional" default="0"/>
</xs:complexType>

```

2.5.21 IProfilerData

Used in

[Patient](#)

Elemente

identification

type:

occurs: 1

description:

Elemente für identification

id

type: string (1..50 Zeichen)

occurs: 1

description:

timeStamp

type: datetime

occurs: 1

description:

counter

type: int

minValue: 1

occurs: 0..1

description:

device

type:
occurs: 0..1
description:

Elemente für device**deviceId**

type: String (1..50 Zeichen)
occurs: 1
description:

softwareVersion

type: String (1..50 Zeichen)
occurs: 1
description:

result

type: [iProfilerResult](#)
use: yes
occurs: 0..10
description:

pupilData

type:
occurs: 0..4
description:

Attribute

side

type: [Sides](#)
use: required
default:
description:

condition

type: [LightingConditions](#)
use: required
default:
description:

Elemente für pupilData**pupilDiameter**

type: float
unity: ?
occurs: 1
description:

pupilCenter

type:
 occurs: 1
 description:

Elemente für pupilCenter

X
 type: float
 unity: ?
 occurs: 1
 description:

Y
 type: float
 unity: ?
 occurs: 1
 description:

application

type: integer
 occurs: 0..1
 description:

XSD

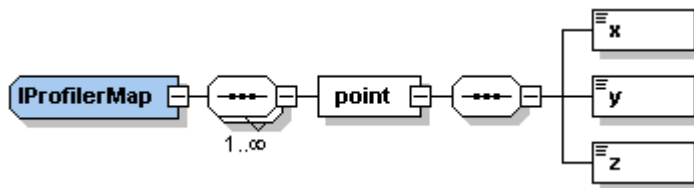
```

<xs:complexType name="IProfilerData">
  <xs:sequence>
    <xs:element name="identification">
      <xs:complexType>
        <xs:sequence>
          <xs:element name="id">
            <xs:simpleType>
              <xs:restriction base="xs:string">
                <xs:minLength value="1"/>
                <xs:maxLength value="50"/>
              </xs:restriction>
            </xs:simpleType>
          </xs:element>
          <xs:element name="timeStamp" type="xs:dateTime"/>
          <xs:element name="counter" minOccurs="0">
            <xs:simpleType>
              <xs:restriction base="xs:int">
                <xs:minInclusive value="0"/>
              </xs:restriction>
            </xs:simpleType>
          </xs:element>
          <xs:element name="device" minOccurs="0">
            <xs:complexType>
              <xs:sequence>
                <xs:element name="deviceId">
                  <xs:simpleType>
                    <xs:restriction
  base="xs:string">

```

```
xs:minLength value="1" />
xs:maxLength value="50" />
softwareVersion">
base="xs:string">
xs:minLength value="1" />
xs:maxLength value="50" />
</xs:restriction>
</xs:simpleType>
</xs:element>
<xs:element name="
</xs:simpleType>
<xs:restriction
</xs:restriction>
</xs:simpleType>
</xs:element>
</xs:sequence>
</xs:complexType>
</xs:element>
</xs:sequence>
</xs:complexType>
</xs:element>
<xs:element name="result" type="IProfilerResult" maxOccurs="2" />
</xs:sequence>
</xs:complexType>
```

2.5.22 IProfilerMap

Used in

[IProfilerResult](#)

Elemente

point

type:

occurs: 1

description:

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X

type: float

unity: μm

occurs: 1

description:

Y

type: float

unity: μm

occurs: 1

description:

Z

type: float

unity: μm

occurs: 1

description:

XSD

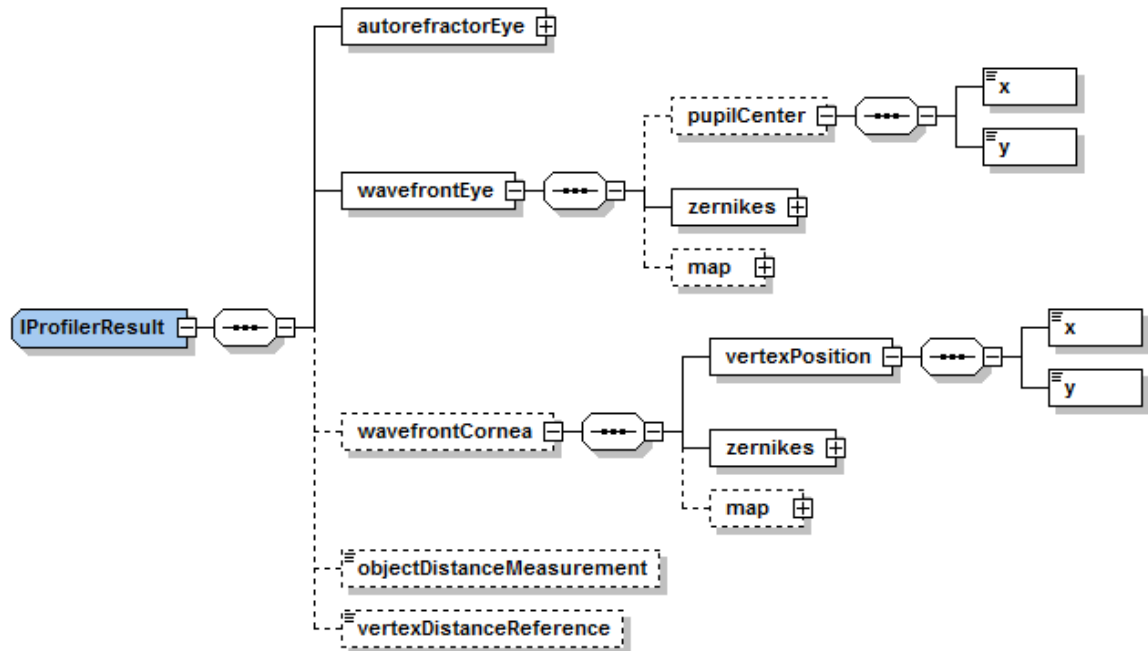
```

<xs:complexType name="IProfilerMap">
  <xs:sequence maxOccurs="unbounded">
    <xs:element name="point">
      <xs:complexType>
        <xs:sequence>
          <xs:element name="x" type="xs:float" />
          <xs:element name="y" type="xs:float" />
          <xs:element name="z" type="xs:float" />
        </xs:sequence>
      </xs:complexType>
    </xs:element>
  </xs:sequence>
</xs:complexType>
  
```

```
        </xs:element>  
    </xs:sequence>  
</xs:complexType>
```

2.5.23 IProfilerResult

Used in
[IProfilerData](#)

Attribute

side

type: [Sides](#)
 use: yes
 description:

Elemente

autorefractorEye

type: [RXDataTypeSimple](#)
 occurs: 1
 description:

wavefrontEye

type:
 occurs: 1
 description:

Elemente für wavefrontEye

pupilCenter

type:
occurs: 1
description:

Elemente für pupilCenter

X
type: float
unity: mm
occurs: 1
description:

Y
type: float
unity: mm
occurs: 1
description:

zernikes
type: Zernikes
occurs: 1
description:

map
type: [IProfilerMap](#)
occurs: 0..1
description:

wavefrontCornea

type:
occurs: 0..1
description:

Elemente für wavefrontCornea

vertexPosition

type:
occurs: 1
description:

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X
type: float
unity: mm
occurs: 1
description:

Y
type: float
unity: mm
occurs: 1

description:

zernikes

type: Zernikes
occurs: 1
description:

map

type: [IProfilerMap](#)
occurs: 1
description:

objectDistanceMeasurement

type: float
unity: ? (todo)
occurs: 0..1
description:

vertexDistanceReference

type: float
unity: ? (todo)
occurs: 0..1
description:

XSD

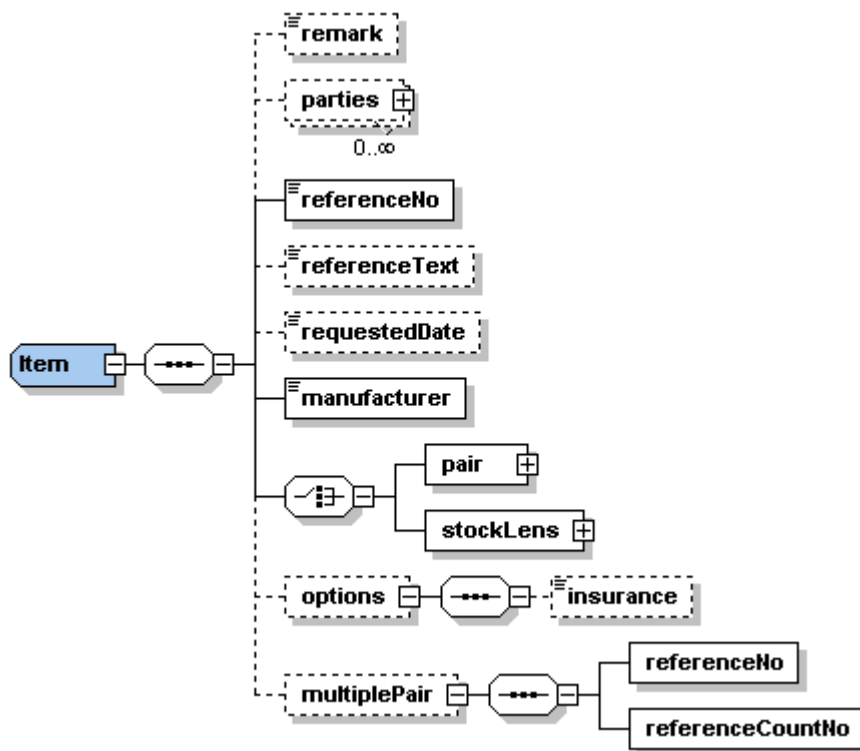
```
<xs:complexType name="IProfilerResult" >
  <xs:sequence>
    <xs:element name="autorefractorEye" type="RefractionSimple" />
    <xs:element name="wavefrontEye" >
      <xs:complexType>
        <xs:sequence>
          <xs:element name="pupilCenter" minOccurs="0" >
            <xs:complexType>
              <xs:sequence>
                <xs:element name="x" type="
xs:float" />
                <xs:element name="y" type="
xs:float" />
              </xs:sequence>
            </xs:complexType>
          </xs:element>
          <xs:element name="zernikes" type="Zernikes" />
          <xs:element name="map" type="IProfilerMap" minOccurs="
0" />
        </xs:sequence>
      </xs:complexType>
    </xs:element>
    <xs:element name="wavefrontCornea" minOccurs="0" >
      <xs:complexType>
        <xs:sequence>
          <xs:element name="vertexPosition" >
            <xs:complexType>
```

```
xs:float"/>
xs:float"/>
0"/>
</xs:sequence>
</xs:complexType>
</xs:element>
<xs:element name="zernikes" type="Zernikes"/>
<xs:element name="map" type="IProfilerMap" minOccurs="
</xs:sequence>
</xs:complexType>
</xs:element>
</xs:sequence>
<xs:attribute name="side" type="Sides" use="required"/>
</xs:complexType>
```

2.5.24 Item

Used in

[b2bOptic](#)

Elemente

remark

type: string

occurs: 0..1

description:

info: CZ: Any comments or remarks are effectively ignored!

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occurs: 0..1

description: buyer's commission

info: CZ: currently ignored - use referenceNo also for textual references (commission)

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multiplePair

type:

occurs: 0..1
description:
info: CZ, Rod :ignored
 Currently only used by ESS.

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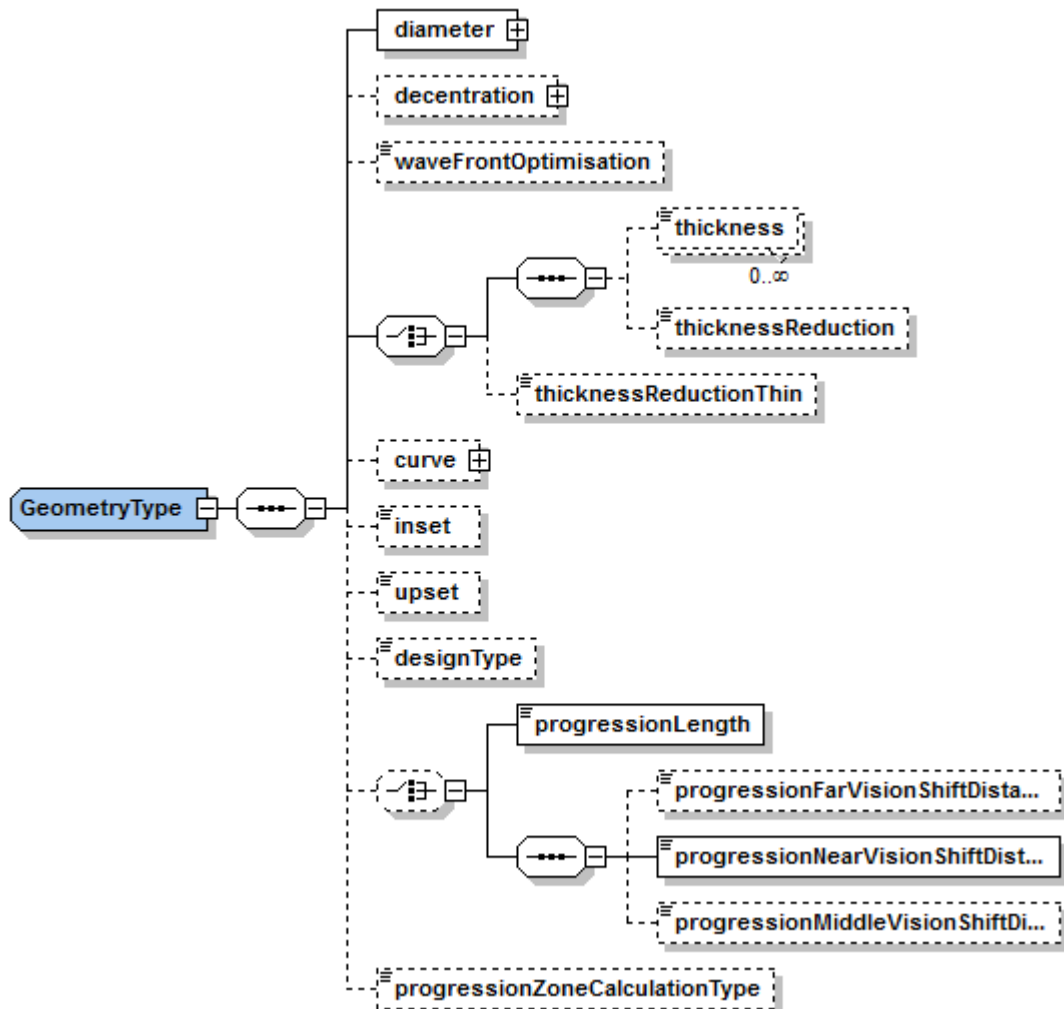
info: CZ: Size correction for the length of progression zone.
ROD: ignored

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info: CZ: Individual gravure text, max five characters. Note: Not all characters supported!

experimental

type: anyType

occurs: 0..n

description:

branding

type: Branding
occurs: 0..1
description: Logo des Herstellers gravieren
info: Currently used only by Essilor and Rodenstock

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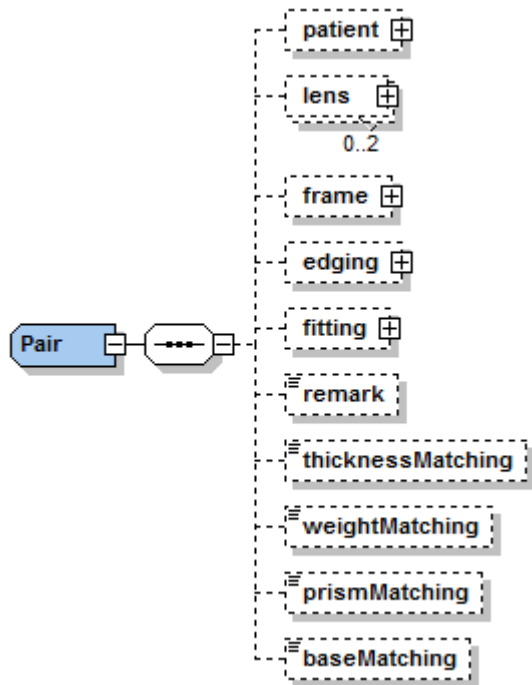
description:

fitting

type: [Fitting](#)
occurs: 0..1
description:
info: CZ: ignored

remark

type: string
occurs: 1
description:
info: CZ: Any comments or remarks are effectively ignored!

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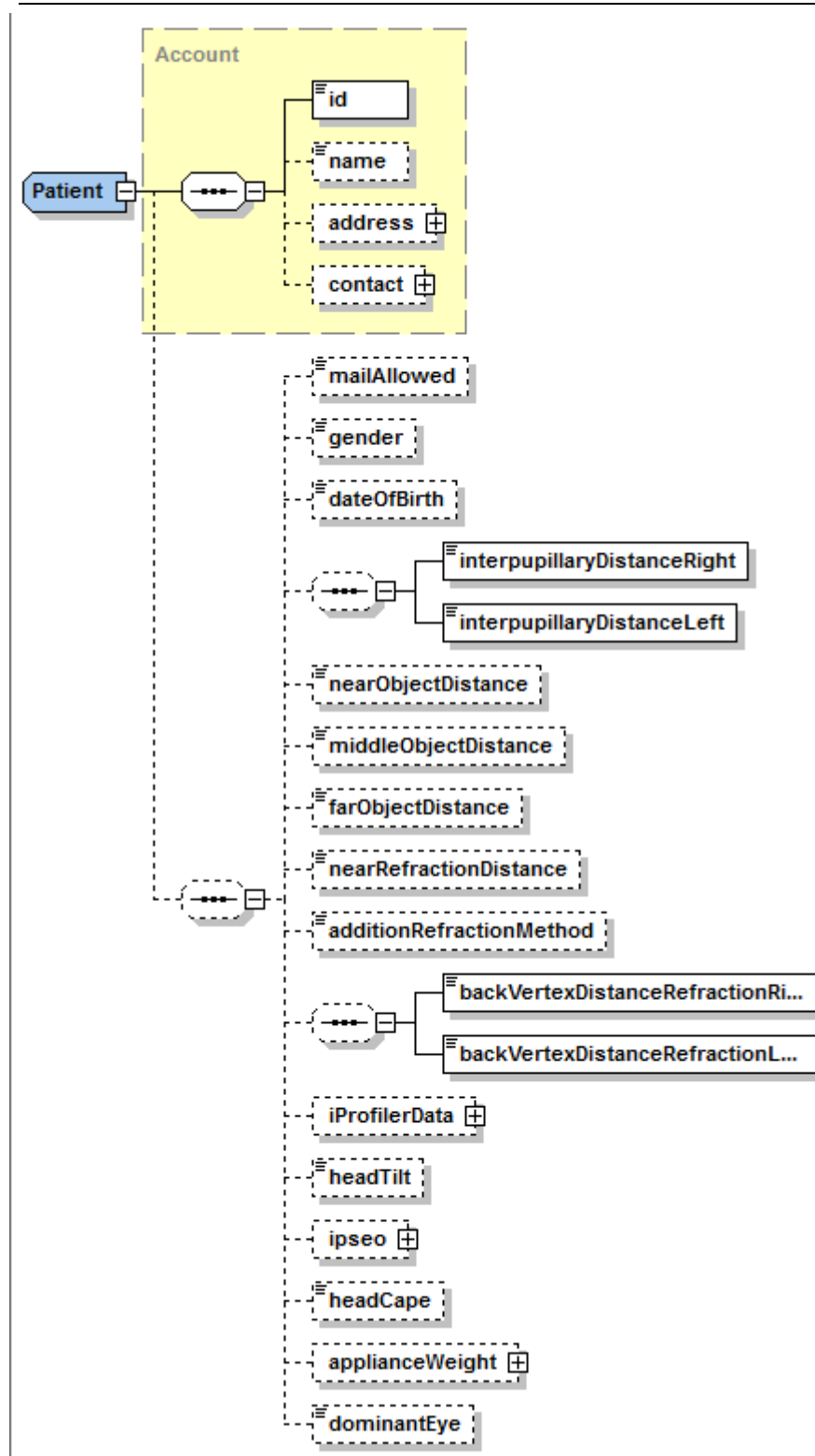
2.5.27 Patient

Used in
[Pair](#)

Remark: /Patient/name should not be used. Instead of, please use /Patient/contact/Lastname

mailAllowed

type: boolean
occurs: 0..1
description:
info: CZ: ignored

gender

type: [Gender](#)
occurs: 0..1
description:
info: CZ: ignored

dateOfBirth

type: dateTime
occurs: 0..1
description:
info: CZ: ignored

interpupillaryDistanceRight

type: float
unity: mm
occurs: 0..1
description: Right PD of Patient. Do not use for centration.
For centration use Element [Centration](#)
info: CZ,SEI: ignored

interpupillaryDistanceLeft

type: float
unity: mm
occurs: 0..1
description: Left PD Patient. Do not use for centration.
For centration use Element [Centration](#)
info: CZ,SEI: ignored

nearObjectDistance

type: float
unity: mm
occurs: 0..1
description:

middleObjectDistance

type: float
unity: mm
occurs: 0..1
description:
info: CZ: ignored

farObjectDistance

type: float
unity: mm
occurs: 0..1
description:
info: CZ: ignored

nearRefractionDistance

type: float
unity: cm
occurs: 0..1
description:
info: CZ: ignored

additionRefractionMethod

type: [AdditionRefractionMethod](#)
occurs: 0..1
description:
info: CZ: ignored

backVertexDistanceRefractionRight

type: float
unity: mm
occurs: 0..1
description:
info: CZ: ignored

backVertexDistanceRefractionLeft

type: float
unity: mm
occurs: 0..1
description:
info: CZ: ignored

iProfilerData

type: [IProfilerData](#)
occurs: 0..1
description:
info: ROD: ignored
SEI: ignored

headTilt

type: float
unity: cm
occurs: 0..1
description:
info: CZ: ignored

ipseo

type: complexType
occurs: 0..1
description: Element for Essilor-Ipseo-Lenses
info: ROD, CZ : ignored

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he

type: float
occurs: 0..1
description: value for Head-Eye-coefficient

st

type: float
occurs: 0..1
description: value for Stab-coefficient

headCape

type: float
unity: degree
occurs: 0..1
description: Element for Essilor - Visiooffice
info: CZ: ignored

applianceWeight

type: [ApplianceWeight](#)
occurs: 0..1
description:
info: CZ: ignored

dominantEye

type: [DominantEyeSide](#)
occurs: 0..1
description:
info:

XSD

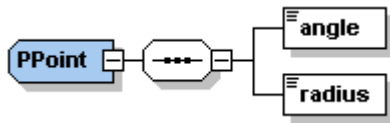
```

<xs:complexType name="Patient">
  <xs:complexContent>
    <xs:extension base="Account">
      <xs:sequence minOccurs="0">
        <xs:element name="mailAllowed" type="xs:boolean" default="false"
minOccurs="0" />
        <xs:element name="gender" type="Gender" minOccurs="0" />
        <xs:element name="dateOfBirth" type="xs:date" minOccurs="0" />
        <xs:sequence minOccurs="0">
          <xs:element name="interpupillaryDistanceRight" type="
xs:float" />
          <xs:element name="interpupillaryDistanceLeft" type="
xs:float" />
        </xs:sequence>
        <xs:element name="nearObjectDistance" type="xs:float" minOccurs
="0" />
        <xs:element name="middleObjectDistance" type="xs:float"
minOccurs="0" />
        <xs:element name="farObjectDistance" type="xs:float" minOccurs="
  
```

```
0" />
minOccurs="0" />
AdditionRefractionMethod" minOccurs="0" />
minOccurs="0" />
0" />
<xs:element name="nearRefractionDistance" type="xs:float"
<xs:element name="additionRefractionMethod" type="
<xs:element name="backVertexDistanceRefraction" type="xs:float"
<xs:element name="iProfilerData" type="IProfilerData" minOccurs="
<xs:element name="headTilt" type="xs:float" minOccurs="0" />
<xs:element name="ipseo" minOccurs="0">
  <xs:complexType>
    <xs:sequence>
      <xs:element name="he" type="xs:float" />
      <xs:element name="st" type="xs:float" />
    </xs:sequence>
  </xs:complexType>
</xs:element>
<xs:element name="headCape" type="xs:float" minOccurs="0" />
<xs:element name="applianceWeight" type="ApplianceWeight"
minOccurs="0" />
  </xs:sequence>
</xs:extension>
</xs:complexContent>
</xs:complexType>
```

2.5.28 PPoint

Used in

[Shape](#)

Elemente

angle

type: float
unity: see dimension
attribute: dimension ([AngleDimension](#))
occurs: 1
description:
info: ROD:possible value for AngleDimension: DEG

radius

type: float
unity: mm
occurs: 1
description:

XSD

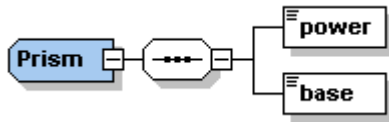
```

<xs:complexType name="PPoint">
  <xs:sequence>
    <xs:element name="angle">
      <xs:complexType>
        <xs:simpleContent>
          <xs:extension base="xs:float">
            AngleDimension" use="optional" default="DEG"/>
          </xs:extension>
        </xs:simpleContent>
      </xs:complexType>
    </xs:element>
    <xs:element name="radius" type="xs:float"/>
  </xs:sequence>
</xs:complexType>
  
```

2.5.29 Prism

Used in

[RXDataType](#)

Elemente

power

type: float
unity: cm/m
occurs: 1
description:

base

type: float
unity: degree
minValue: 0.0
maxValue: 360.0
occurs: 1
description:

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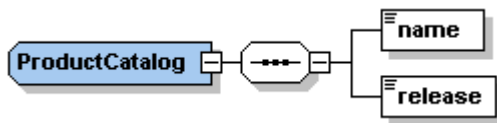
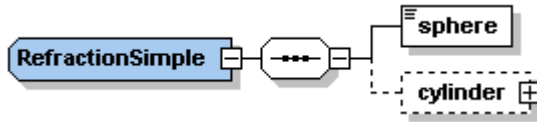
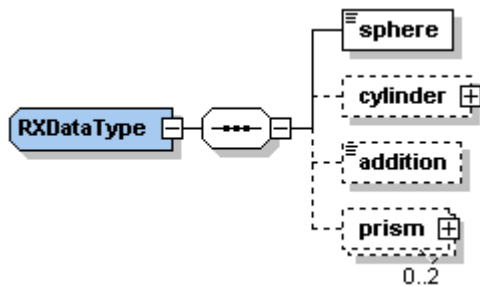
Elemente

sphere

type: float
 unity: dpt
 occurs: 1
 description:

cylinder

type: [Cylinder](#)
 occurs: 0..1
 description:

addition

type: float
 unity: dpt
 minValue: 0.25
 occurs: 0..1
 description:

prism

type: [Prism](#)
 occurs: 0..2
 description:

XSD

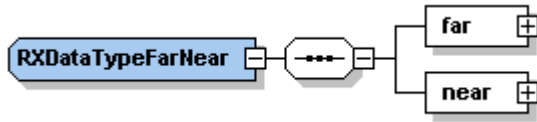
```

<xs:complexType name="RXDataType">
  <xs:sequence>
    <xs:element name="sphere">
      <xs:simpleType>
        <xs:restriction base="xs:float">
          <xs:minInclusive value="-50"/>
          <xs:maxInclusive value="50"/>
        </xs:restriction>
      </xs:simpleType>
    </xs:element>
    <xs:element name="cylinder" type="Cylinder" minOccurs="0"/>
  
```

```
<xs:element name="addition" minOccurs="0">
  <xs:simpleType>
    <xs:restriction base="xs:float">
      <xs:minInclusive value="0.25" />
    </xs:restriction>
  </xs:simpleType>
</xs:element>
<xs:element name="prism" type="Prism" minOccurs="0" maxOccurs="2" />
</xs:sequence>
</xs:complexType>
```

2.5.33 RxDataTypeFarNear

Used in

[Lens](#)

Attribute

reference

type: [RxDataReferences](#)

use: required

description:

Elemente

far

type: [RxDataTypeFn](#)

occurs: 1

description:

near

type: [RxDataTypeFn](#)

occurs: 1

description:

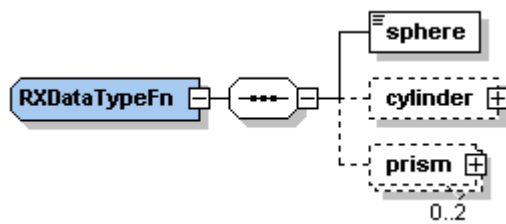
XSD

```
<xs:complexType name="RXDataTypeFarNear">
  <xs:sequence>
    <xs:element name="far" type="RxDataTypeFn"/>
    <xs:element name="near" type="RxDataTypeFn"/>
  </xs:sequence>
  <xs:attribute name="reference" type="RxDataReferences" use="required"/>
</xs:complexType>
```

2.5.34 RxDataTypeFn

Used in

[RxDataTypeFarNear](#)

Elemente

sphere

type: float
unity: dpt
occurs: 1
description:

cylinder

type: [Cylinder](#)
occurs: 0..1
description:

prism

type: [Prism](#)
occurs: 0..2
description:

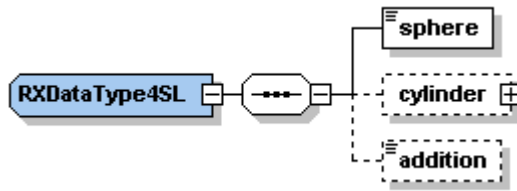
XSD

```
<xs:complexType name="RxDataTypeFn">
  <xs:sequence>
    <xs:element name="sphere" type="xs:float"/>
    <xs:element name="cylinder" type="Cylinder" minOccurs="0"/>
    <xs:element name="prism" type="Prism" minOccurs="0" maxOccurs="2"/>
  </xs:sequence>
</xs:complexType>
```

2.5.35 RXDataType4SL

Used in

[StockLens](#)

Elemente

sphere

type: float
unity: dpt
minValue: -50
maxValue: 50
occurs: 1
description:

cylinder

type: [Cylinder](#)
occurs: 0..1
description:

addition

type: float
unity: dpt
minValue: 0.25
occurs: 0..1
description:

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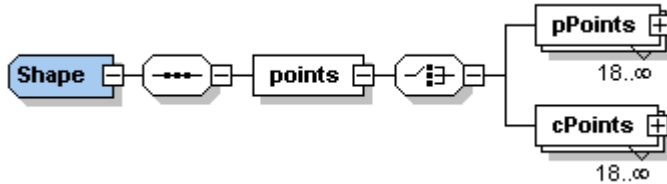
description:

info:

CZ: In an order (header[@msgType=ORDER]) any number of polar coordinates other than 36 will result in an exception thrown to the client.

cPoints

type: [cPoint](#)

occurs: 18..n

description:

info:

CZ: currently ignored - use polar coordinates instead!

XSD

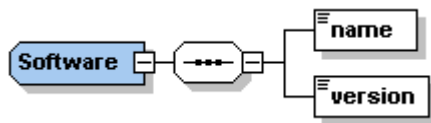
```

<xs:complexType name="Shape">
  <xs:sequence>
    <xs:element name="points">
      <xs:complexType>
        <xs:choice>
          <xs:element name="pPoints" type="PPoint" minOccurs="18"
" maxOccurs="unbounded"/>
          <xs:element name="cPoints" type="CPoint" minOccurs="18"
" maxOccurs="unbounded"/>
        </xs:choice>
      </xs:complexType>
    </xs:element>
  </xs:sequence>
</xs:complexType>
  
```

2.5.37 Software

Used in

[Header](#)

Attribute

typeOf

type: [SoftwareTypes](#)

use: required

description:

Elemente

name

type: string

occurs: 1

description:

version

type: string

occurs: 1

description:

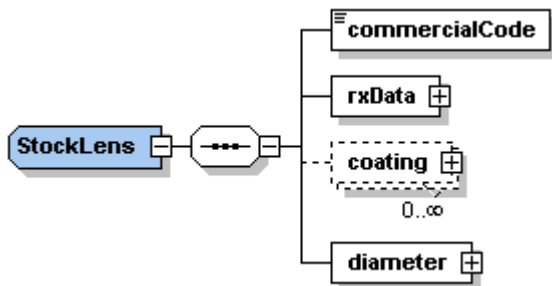
XSD

```
<xs:complexType name="Software">
  <xs:sequence>
    <xs:element name="name" type="xs:string"/>
    <xs:element name="version" type="xs:string"/>
  </xs:sequence>
  <xs:attribute name="typeOf" type="SoftwareTypes" use="required"/>
</xs:complexType>
```

2.5.38 StockLens

Used in

[Pair Item](#)

Attribute

quantity

type: integer
use: required
description:

side

type: [Sides](#)
use: required
description:

Elemente

commercialCode

type: string
occurs: 1
description:

rxData

type: [RXDataType4SL](#)
occurs: 1
description:

coating

type: [Coating](#)
occurs: 0..n
description:

diameter

type: Diameter
occurs: 1
description:

XSD

```
<xs:complexType name="StockLens">
  <xs:sequence>
    <xs:element name="commercialCode" type="xs:string"/>
    <xs:element name="rxData" type="RXDataType4SL"/>
    <xs:element name="coating" type="Coating" minOccurs="0" maxOccurs="unbounded" />
  </xs:sequence>
  <xs:element name="diameter" type="Diameter" />
  <xs:attribute name="quantity" type="xs:integer" use="required"/>
  <xs:attribute name="side" type="Sides" use="required"/>
</xs:complexType>
```

2.5.39 TimeStamps

Used in

[Header](#)

Elemente

dateTime

type: dateTime
attribute: step (Type [MsgSteps](#))
occurs: 1
description:

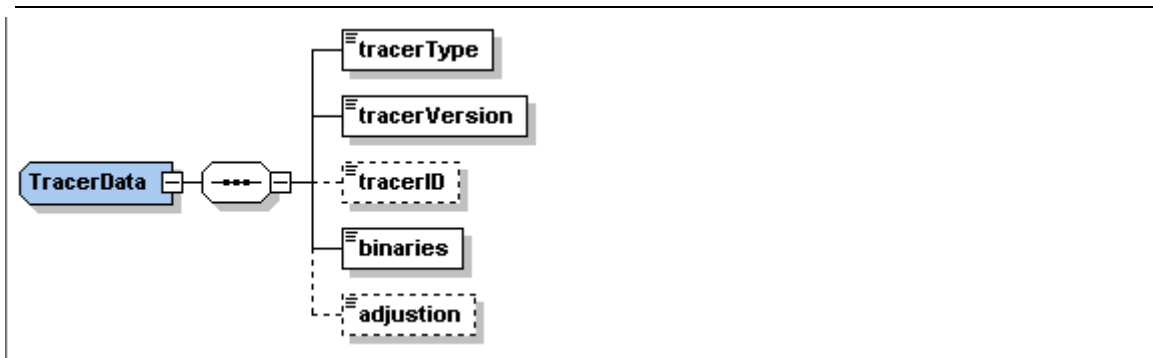
XSD

```

<xs:complexType name="TimeStamps">
  <xs:sequence>
    <xs:element name="dateTime">
      <xs:complexType>
        <xs:simpleContent>
          <xs:extension base="xs:dateTime">
            <xs:attribute name="step" type="MsgSteps" use="
required" />
          </xs:extension>
        </xs:simpleContent>
      </xs:complexType>
    </xs:element>
  </xs:sequence>
</xs:complexType>
  
```

2.5.40 TracerData

Used in
[Frame](#)

Elemente

tracerType

type: [TracerType](#)
 occurs: 1
 description:

tracerVersion

type: string
 occurs: 1
 description:
 info: CZ: ignored

TracerId

type: string
 occurs: 0..1
 description: Serialnumber of tracer

binaries

type: hexBinary
 attribute: format (Type [TracerBinaryFormat](#))
 occurs: 1
 description:

adjustion

type: float
 unity: mm
 occurs: 0..1
 description: general size adjustment parameter
 info: CZ: Size correction in the Nidek data set.
 ROD: ignored

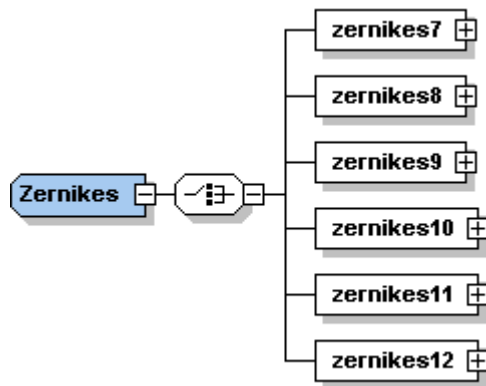
XSD

```
<xs:complexType name="TracerData" >
  <xs:sequence>
    <xs:element name="tracerType" type="TracerType" />
    <xs:element name="tracerVersion" type="xs:string" />
    <xs:element name="tracerID" type="xs:string" minOccurs="0" />
    <xs:element name="binaries" >
      <xs:complexType>
        <xs:simpleContent>
          <xs:extension base="xs:hexBinary" >
            <xs:attribute name="format" type="TracerBinaryFormat" />
          </xs:extension>
        </xs:simpleContent>
      </xs:complexType>
    </xs:element>
    <xs:element name="adjustment" type="xs:float" minOccurs="0" />
  </xs:sequence>
</xs:complexType>
```

2.5.41 Zernikes

Used in

[IProfilerResult](#)

Elemente

zernikes7

type: [Zernikes7](#)
occurs: 1
description:

zernikes8

type: [Zernikes8](#)
occurs: 1
description:

zernikes9

type: [Zernikes9](#)
occurs: 1
description:

zernikes10

type: [Zernikes10](#)
occurs: 1
description:

zernikes11

type: [Zernikes11](#)
occurs: 1
description:

zernikes12

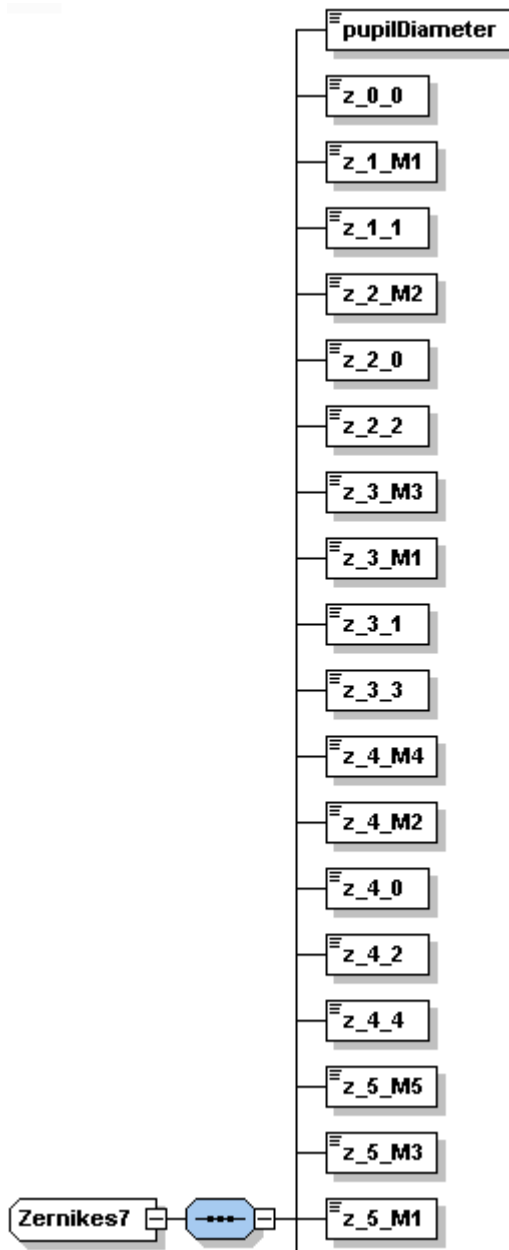
type: [Zernikes12](#)
occurs: 1
description:

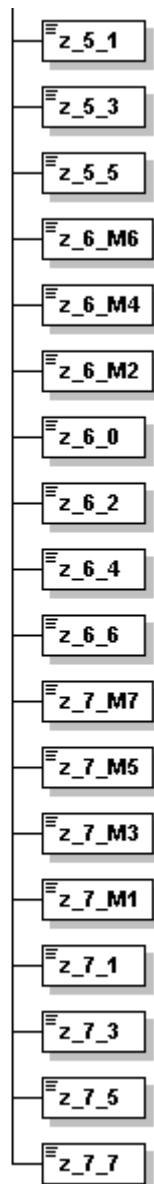
XSD

```
<xs:complexType name="Zernikes">
  <xs:choice>
    <xs:element name="zernikes7" type="Zernikes7"/>
    <xs:element name="zernikes8" type="Zernikes8"/>
    <xs:element name="zernikes9" type="Zernikes9"/>
    <xs:element name="zernikes10" type="Zernikes10"/>
    <xs:element name="zernikes11" type="Zernikes11"/>
    <xs:element name="zernikes12" type="Zernikes12"/>
  </xs:choice>
</xs:complexType>
```

2.5.42 Zernikes7

Used in
[Zernikes](#)

Elemente

pupilDiameter

type: float
unity: mm
occurs: 1
description:

z_1_M1

type: float
unity: μm
occurs: 1
description:

z_1_1

type: float
unity: μm
occurs: 1
description:

z 2 M2

type: float
unity: μm
occurs: 1
description:

z 2 0

type: float
unity: μm
occurs: 1
description:

z 2 2

type: float
unity: μm
occurs: 1
description:

z 3 M3

type: float
unity: μm
occurs: 1
description:

z 3 M1

type: float
unity: μm
occurs: 1
description:

z 3 1

type: float
occurs: 1
unity: μm
description:

z 3 3

type: float
occurs: 1
unity: μm
description:

z 4 M4

type: float
unity: μm
occurs: 1
description:

z 4 M2

type: float
unity: μm
occurs: 1
description:

z 4 0

type: float
unity: μm
occurs: 1
description:

z 4 2

type: float
unity: μm
occurs: 1
description:

z 4 4

type: float
unity: μm
occurs: 1
description:

z 5 M5

type: float
unity: μm
occurs: 1
description:

z 5 M3

type: float
unity: μm
occurs: 1
description:

z 5 M1

type: float
unity: μm
occurs: 1
description:

z 5 1

type: float
unity: μm
occurs: 1
description:

z 5 3

type: float
unity: μm
occurs: 1
description:

z 5 5

type: float
unity: μm
occurs: 1
description:

z 6 M6

type: float
unity: μm
occurs: 1
description:

z 6 M4

type: float
unity: μm
occurs: 1
description:

z 6 M2

type: float
unity: μm
occurs: 1
description:

z 6 0

type: float
unity: μm
occurs: 1
description:

z 6 2

type: float
unity: μm
occurs: 1
description:

z 6 4

type: float
unity: μm
occurs: 1
description:

z 6 6

type: float
unity: μm
occurs: 1
description:

z 7 M7

type: float
unity: μm
occurs: 1
description:

z 7 M5

type: float
unity: μm
occurs: 1
description:

z 7 M3

type: float
unity: μm
occurs: 1
description:

z 7 M1

type: float
unity: μm
occurs: 1
description:

z 7 1

type: float
unity: μm
occurs: 1
description:

z 7 3

type: float
unity: μm
occurs: 1
description:

z 7 5

type: float
unity: μm
occurs: 1
description:

z 7 7

type: float
unity: μm
occurs: 1
description:

XSD

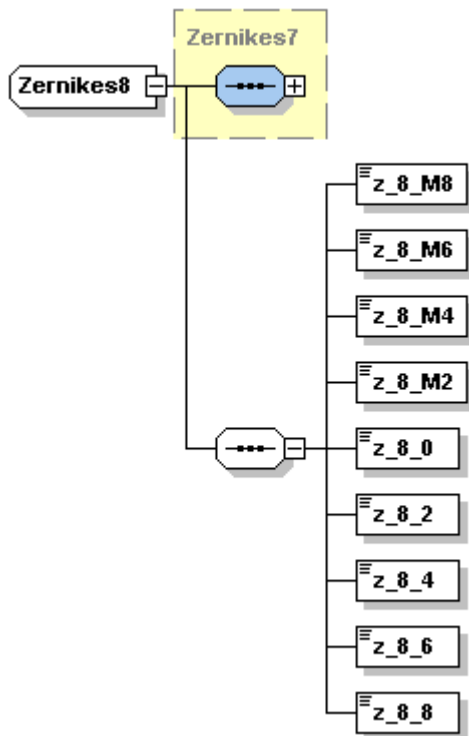
```
<xs:complexType name="Zernikes7">  
  <xs:sequence>  
    <xs:element name="pupilDiameter" type="xs:float"/>  
    <xs:element name="z_0_0" type="xs:float"/>  
    <xs:element name="z_1_M1" type="xs:float"/>  
    <xs:element name="z_1_1" type="xs:float"/>  
    <xs:element name="z_2_M2" type="xs:float"/>  
    <xs:element name="z_2_0" type="xs:float"/>  
    <xs:element name="z_2_2" type="xs:float"/>  
    <xs:element name="z_3_M3" type="xs:float"/>  
    <xs:element name="z_3_M1" type="xs:float"/>  
  </xs:sequence>  
</xs:complexType>
```

```
<xs:element name="z_3_1" type="xs:float" />
<xs:element name="z_3_3" type="xs:float" />
<xs:element name="z_4_M4" type="xs:float" />
<xs:element name="z_4_M2" type="xs:float" />
<xs:element name="z_4_0" type="xs:float" />
<xs:element name="z_4_2" type="xs:float" />
<xs:element name="z_4_4" type="xs:float" />
<xs:element name="z_5_M5" type="xs:float" />
<xs:element name="z_5_M3" type="xs:float" />
<xs:element name="z_5_M1" type="xs:float" />
<xs:element name="z_5_1" type="xs:float" />
<xs:element name="z_5_3" type="xs:float" />
<xs:element name="z_5_5" type="xs:float" />
<xs:element name="z_6_M6" type="xs:float" />
<xs:element name="z_6_M4" type="xs:float" />
<xs:element name="z_6_M2" type="xs:float" />
<xs:element name="z_6_0" type="xs:float" />
<xs:element name="z_6_2" type="xs:float" />
<xs:element name="z_6_4" type="xs:float" />
<xs:element name="z_6_6" type="xs:float" />
<xs:element name="z_7_M7" type="xs:float" />
<xs:element name="z_7_M3" type="xs:float" />
<xs:element name="z_7_M1" type="xs:float" />
<xs:element name="z_7_M5" type="xs:float" />
<xs:element name="z_7_1" type="xs:float" />
<xs:element name="z_7_3" type="xs:float" />
<xs:element name="z_7_5" type="xs:float" />
<xs:element name="z_7_7" type="xs:float" />
</xs:sequence>
</xs:complexType>
```

2.5.43 Zernikes8

Used in

[Zernikes](#)

Elemente

z_8_M8

type: float
unity: μm
occurs: 1
description:

z_8_M6

type: float
unity: μm
occurs: 1
description:

z_8_M4

type: float
unity: μm
occurs: 1
description:

z_8_M2

type: float

unity: μm
 occurs: 1
 description:

z 8 0

type: float
 unity: μm
 occurs: 1
 description:

z 8 2

type: float
 unity: μm
 occurs: 1
 description:

z 8 4

type: float
 unity: μm
 occurs: 1
 description:

z 8 6

type: float
 unity: μm
 occurs: 1
 description:

z 8 8

type: float
 unity: μm
 occurs: 1
 description:

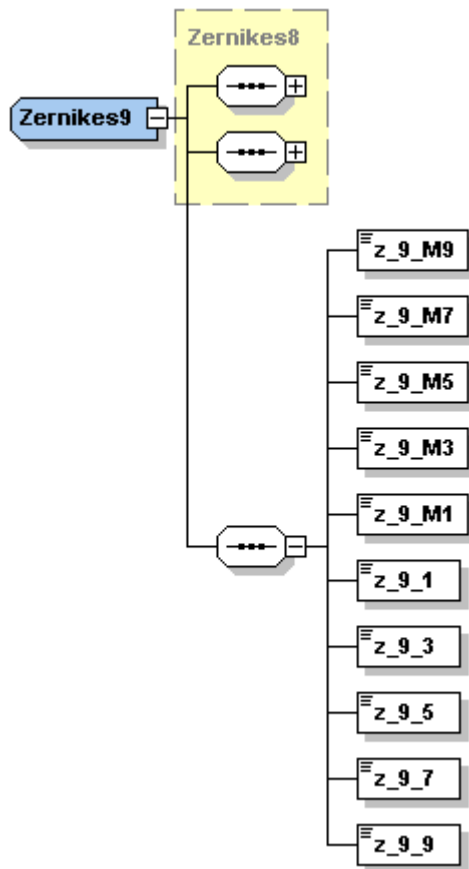
XSD

```
<xs:complexType name="Zernikes8">
  <xs:complexContent>
    <xs:extension base="Zernikes7">
      <xs:sequence>
        <xs:element name="z_8_M8" type="xs:float"/>
        <xs:element name="z_8_M6" type="xs:float"/>
        <xs:element name="z_8_M4" type="xs:float"/>
        <xs:element name="z_8_M2" type="xs:float"/>
        <xs:element name="z_8_0" type="xs:float"/>
        <xs:element name="z_8_2" type="xs:float"/>
        <xs:element name="z_8_4" type="xs:float"/>
        <xs:element name="z_8_6" type="xs:float"/>
        <xs:element name="z_8_8" type="xs:float"/>
      </xs:sequence>
    </xs:extension>
  </xs:complexContent>
</xs:complexType>
```

2.5.44 Zernikes9

Used in

[Zernikes](#)

Elemente

z_9_M9

type: float
unity: μm
occurs: 1
description:

z_9_M7

type: float
unity: μm
occurs: 1
description:

z_9_M5

type: float
unity: μm
occurs: 1

description:

z_9_M3

type: float
unity: μm
occurs: 1
description:

z_9_M1

type: float
unity: μm
occurs: 1
description:

z_9_1

type: float
unity: μm
occurs: 1
description:

z_9_3

type: float
unity: μm
occurs: 1
description:

z_9_5

type: float
unity: μm
occurs: 1
description:

z_9_7

type: float
unity: μm
occurs: 1
description:

z_9_9

type: float
unity: μm
occurs: 1
description:

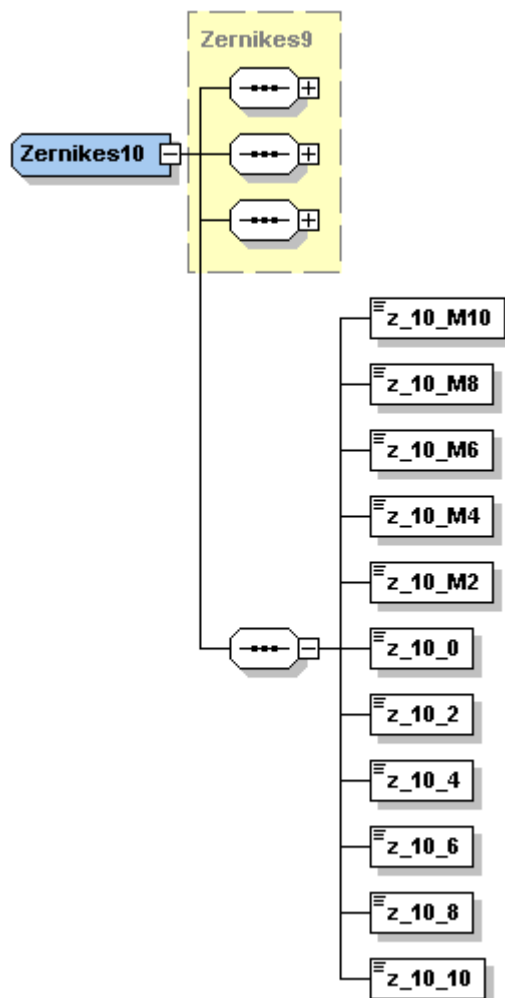
XSD

```
<xs:complexType name="Zernikes9">  
  <xs:complexContent>  
    <xs:extension base="Zernikes8">  
      <xs:sequence>  
        <xs:element name="z_9_M9" type="xs:float"/>  
        <xs:element name="z_9_M7" type="xs:float"/>  
      </xs:sequence>  
    </xs:extension>  
  </xs:complexContent>  
</xs:complexType>
```

```
<xs:element name="z_9_M5" type="xs:float" />
<xs:element name="z_9_M3" type="xs:float" />
<xs:element name="z_9_M1" type="xs:float" />
<xs:element name="z_9_1" type="xs:float" />
<xs:element name="z_9_3" type="xs:float" />
<xs:element name="z_9_5" type="xs:float" />
<xs:element name="z_9_7" type="xs:float" />
<xs:element name="z_9_9" type="xs:float" />
</xs:sequence>
</xs:extension>
</xs:complexContent>
</xs:complexType>
```

2.5.45 Zernikes10

Used in
[Zernikes](#)

Elemente

z_10_M10

type: float
unity: μm
occurs: 1
description:

z_10_M8

type: float
unity: μm
occurs: 1
description:

z 10 M6

type: float
unity: μm
occurs: 1
description:

z 10 M4

type: float
unity: μm
occurs: 1
description:

z 10 M2

type: float
unity: μm
occurs: 1
description:

z 10 0

type: float
unity: μm
occurs: 1
description:

z 10 2

type: float
unity: μm
occurs: 1
description:

z 10 4

type: float
unity: μm
occurs: 1
description:

z 10 6

type: float
unity: μm
occurs: 1
description:

z 10 8

type: float
unity: μm
occurs: 1
description:

z 10 10

type: float
unity: μm
occurs: 1

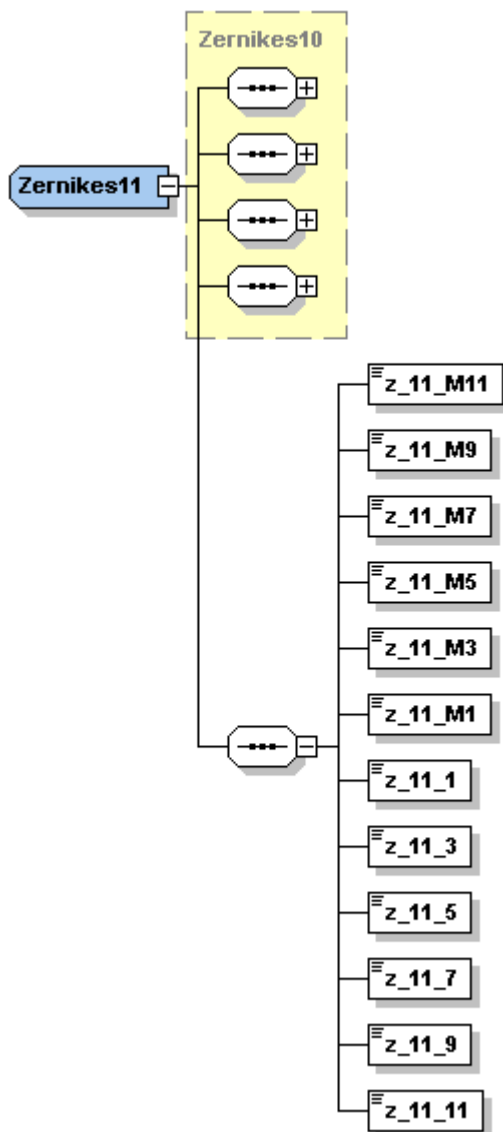
description:

XSD

```
<xs:complexType name="Zernikes10">
  <xs:complexContent>
    <xs:extension base="Zernikes9">
      <xs:sequence>
        <xs:element name="z_10_M10" type="xs:float"/>
        <xs:element name="z_10_M8" type="xs:float"/>
        <xs:element name="z_10_M6" type="xs:float"/>
        <xs:element name="z_10_M4" type="xs:float"/>
        <xs:element name="z_10_M2" type="xs:float"/>
        <xs:element name="z_10_0" type="xs:float"/>
        <xs:element name="z_10_2" type="xs:float"/>
        <xs:element name="z_10_4" type="xs:float"/>
        <xs:element name="z_10_6" type="xs:float"/>
        <xs:element name="z_10_8" type="xs:float"/>
        <xs:element name="z_10_10" type="xs:float"/>
      </xs:sequence>
    </xs:extension>
  </xs:complexContent>
</xs:complexType>
```

2.5.46 Zernikes11

Used in
[Zernikes](#)

Elemente

z_11_M11

type: float
 unity: μm
 occurs: 1
 description:

z_11_M9

type: float
unity: μm
occurs: 1
description:

z 11 M7

type: float
unity: μm
occurs: 1
description:

z 11 M5

type: float
unity: μm
occurs: 1
description:

z 11 M3

type: float
unity: μm
occurs: 1
description:

z 11 M1

type: float
unity: μm
occurs: 1
description:

z 11 1

type: float
unity: μm
occurs: 1
description:

z 11 3

type: float
unity: μm
occurs: 1
description:

z 11 5

type: float
unity: μm
occurs: 1
description:

z 11 7

type: float
unity: μm
occurs: 1
description:

z_11_9

type: float
unity: μm
occurs: 1
description:

z_11_11

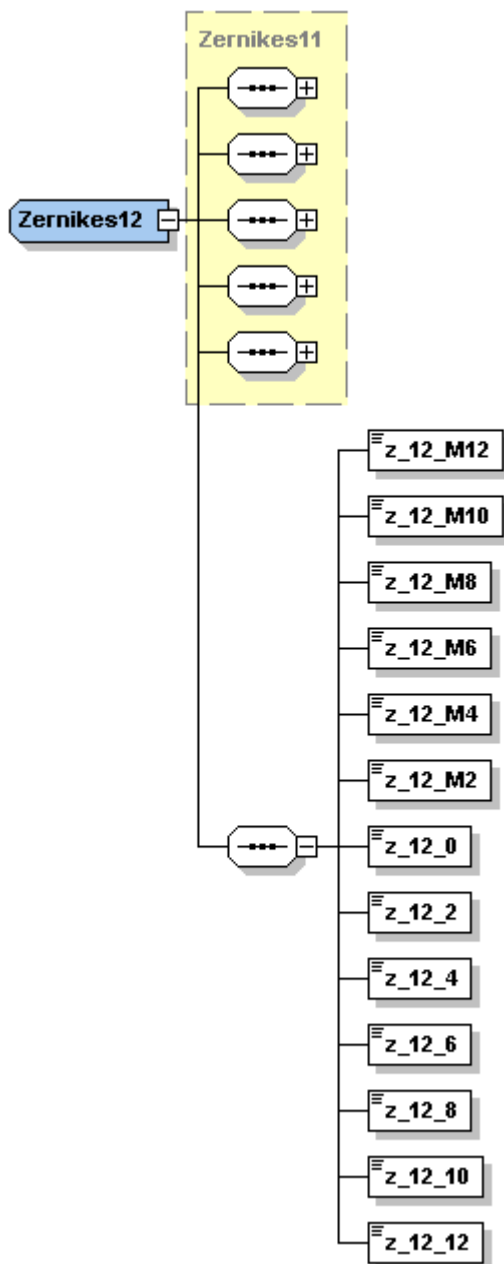
type: float
unity: μm
occurs: 1
description:

XSD

```
<xs:complexType name="Zernikes11">
  <xs:complexContent>
    <xs:extension base="Zernikes10">
      <xs:sequence>
        <xs:element name="z_11_M11" type="xs:float"/>
        <xs:element name="z_11_M9" type="xs:float"/>
        <xs:element name="z_11_M7" type="xs:float"/>
        <xs:element name="z_11_M5" type="xs:float"/>
        <xs:element name="z_11_M3" type="xs:float"/>
        <xs:element name="z_11_M1" type="xs:float"/>
        <xs:element name="z_11_1" type="xs:float"/>
        <xs:element name="z_11_3" type="xs:float"/>
        <xs:element name="z_11_5" type="xs:float"/>
        <xs:element name="z_11_7" type="xs:float"/>
        <xs:element name="z_11_9" type="xs:float"/>
        <xs:element name="z_11_11" type="xs:float"/>
      </xs:sequence>
    </xs:extension>
  </xs:complexContent>
</xs:complexType>
```

2.5.47 Zernikes12

Used in
[Zernikes](#)

Elemente

z_12_M12

type: float
unity: μm

occurs: 1
description:

z 12 M10

type: float
unity: μm
occurs: 1
description:

z 12 M8

type: float
unity: μm
occurs: 1
description:

z 12 M6

type: float
unity: μm
occurs: 1
description:

z 12 M4

type: float
unity: μm
occurs: 1
description:

z 12 M2

type: float
unity: μm
occurs: 1
description:

z 12 0

type: float
unity: μm
occurs: 1
description:

z 12 2

type: float
unity: μm
occurs: 1
description:

z 12 4

type: float
unity: μm

occurs: 1
description:

z_12_6

type: float
unity: μm
occurs: 1
description:

z_12_8

type: float
unity: μm
occurs: 1
description:

z_12_10

type: float
unity: μm
occurs: 1
description:

z_12_12

type: float
unity: μm
occurs: 1
description:

XSD

```
<xs:complexType name="Zernikes12">
  <xs:complexContent>
    <xs:extension base="Zernikes11">
      <xs:sequence>
        <xs:element name="z_12_M12" type="xs:float"/>
        <xs:element name="z_12_M10" type="xs:float"/>
        <xs:element name="z_12_M8" type="xs:float"/>
        <xs:element name="z_12_M6" type="xs:float"/>
        <xs:element name="z_12_M4" type="xs:float"/>
        <xs:element name="z_12_M2" type="xs:float"/>
        <xs:element name="z_12_0" type="xs:float"/>
        <xs:element name="z_12_2" type="xs:float"/>
        <xs:element name="z_12_4" type="xs:float"/>
        <xs:element name="z_12_6" type="xs:float"/>
        <xs:element name="z_12_8" type="xs:float"/>
        <xs:element name="z_12_10" type="xs:float"/>
        <xs:element name="z_12_12" type="xs:float"/>
      </xs:sequence>
    </xs:extension>
  </xs:complexContent>
</xs:complexType>
```

Chapter

3 XSD - Versionen

3.1 Version 1.2.3

```
<?xml version="1.0" encoding="UTF-8"?>
<!-- edited with XML Spy v4.4 U (http://www.xmlspy.com) by Carl Zeiss (Carl Zeiss) -->
<xs:schema xmlns:xs="http://www.w3.org/2001/XMLSchema" elementFormDefault="qualified"
attributeFormDefault="unqualified" version="1.2.3">
  <xs:element name="b2bOptic">
    <xs:annotation>
      <xs:documentation/>
    </xs:annotation>
    <xs:complexType>
      <xs:sequence>
        <xs:element name="header" type="Header"/>
        <xs:element name="items">
          <xs:complexType>
            <xs:sequence>
              <xs:element name="item" type="Item"
maxOccurs="unbounded"/>
            </xs:sequence>
          </xs:complexType>
        </xs:element>
        <xs:element name="experimental" type="xs:anyType" minOccurs
="0" maxOccurs="unbounded"/>
      </xs:sequence>
    </xs:complexType>
  </xs:element>
  <xs:simpleType name="MsgTypes">
    <xs:restriction base="xs:string">
      <xs:enumeration value="ORDER"/>
      <xs:enumeration value="REQUEST"/>
      <xs:enumeration value="CALCULATION"/>
      <xs:enumeration value="VALIDATION"/>
    </xs:restriction>
  </xs:simpleType>
  <xs:simpleType name="BevelPosType">
    <xs:restriction base="xs:string">
      <xs:enumeration value="AUTO"/>
      <xs:enumeration value="FRONT"/>
      <xs:enumeration value="BACK"/>
      <xs:enumeration value="RELATION"/>
    </xs:restriction>
  </xs:simpleType>
  <xs:simpleType name="BevelTypes">
    <xs:restriction base="xs:string">
      <xs:enumeration value="NORMAL"/>
      <xs:enumeration value="FLAT"/>
      <xs:enumeration value="GROOVED"/>
    </xs:restriction>
  </xs:simpleType>
  <xs:simpleType name="SoftwareTypes">
    <xs:restriction base="xs:string">
      <xs:enumeration value="ORIGINATOR"/>
      <xs:enumeration value="VERIFIER"/>
      <xs:enumeration value="SENDER"/>
    </xs:restriction>
  </xs:simpleType>
  <xs:simpleType name="MsgStates">
    <xs:restriction base="xs:string">
      <xs:enumeration value="NEW"/>
      <xs:enumeration value="UPDATE"/>
      <xs:enumeration value="CANCEL"/>
    </xs:restriction>
  </xs:simpleType>
  <xs:simpleType name="MsgSteps">
    <xs:restriction base="xs:string">
      <xs:enumeration value="CREATE"/>
      <xs:enumeration value="TRANSFER"/>
      <xs:enumeration value="RECEIPT"/>
    </xs:restriction>
  </xs:simpleType>
  <xs:simpleType name="Roles">
```

```

        <xs:restriction base="xs:string">
            <xs:enumeration value="ORIGINATOR" />
            <xs:enumeration value="SUPPLIER" />
            <xs:enumeration value="SHIPTO" />
            <xs:enumeration value="CARRIER" />
            <xs:enumeration value="INVOICETO" />
            <xs:enumeration value="PAIDBY" />
            <xs:enumeration value="MANUFACTURER" />
        </xs:restriction>
    </xs:simpleType>
    <xs:simpleType name="FrameMaterials">
        <xs:restriction base="xs:string">
            <xs:enumeration value="METAL" />
            <xs:enumeration value="PLASTIC" />
            <xs:enumeration value="OPTYL" />
            <xs:enumeration value="NYLOR" />
            <xs:enumeration value="DRILLED" />
            <xs:enumeration value="SPECIAL" />
        </xs:restriction>
    </xs:simpleType>
    <xs:simpleType name="HeightReferences">
        <xs:restriction base="xs:string">
            <xs:enumeration value="OVERBOX" />
            <xs:enumeration value="OVERSHAPE" />
        </xs:restriction>
    </xs:simpleType>
    <xs:simpleType name="OCReferences">
        <xs:restriction base="xs:string">
            <xs:enumeration value="FAR" />
            <xs:enumeration value="NEAR" />
        </xs:restriction>
    </xs:simpleType>
    <xs:simpleType name="CoatingTypes">
        <xs:restriction base="xs:string">
            <xs:enumeration value="COLOR" />
            <xs:enumeration value="UV" />
            <xs:enumeration value="ANTIREFLEX" />
            <xs:enumeration value="HARD" />
            <xs:enumeration value="CLEAN" />
            <xs:enumeration value="OTHER" />
        </xs:restriction>
    </xs:simpleType>
    <xs:simpleType name="EdgingType">
        <xs:restriction base="xs:string">
            <xs:enumeration value="NONE" />
            <xs:enumeration value="ROUGHING" />
            <xs:enumeration value="ONSHAPE" />
            <xs:enumeration value="GIVENFRAME" />
            <xs:enumeration value="ORDEREDFRAME" />
        </xs:restriction>
    </xs:simpleType>
    <xs:simpleType name="AngleDimension">
        <xs:restriction base="xs:string">
            <xs:enumeration value="DEG" />
            <xs:enumeration value="RAD" />
            <xs:enumeration value="MM" />
        </xs:restriction>
    </xs:simpleType>
    <xs:simpleType name="Currencies">
        <xs:restriction base="xs:string">
            <xs:length value="3" />
            <xs:enumeration value="EUR" />
            <xs:enumeration value="USD" />
            <xs:enumeration value="SFR" />
        </xs:restriction>
    </xs:simpleType>
    <xs:simpleType name="Sides">
        <xs:restriction base="xs:string">
            <xs:enumeration value="RIGHT" />
            <xs:enumeration value="LEFT" />
            <xs:enumeration value="UNDEFINED" />
        </xs:restriction>
    </xs:simpleType>
    <xs:simpleType name="ThicknessReferences">
        <xs:restriction base="xs:string">

```

```

        <xs:enumeration value="REDUCEWITHSHAPE" />
        <xs:enumeration value="CENTER" />
        <xs:enumeration value="EDGE" />
        <xs:enumeration value="DRILLHOLE" />
        <xs:enumeration value="FORNYLOR" />
        <xs:enumeration value="THIN" />
    </xs:restriction>
</xs:simpleType>
<xs:simpleType name="DeliveryMethods">
    <xs:restriction base="xs:string" />
</xs:simpleType>
<xs:simpleType name="References">
    <xs:restriction base="xs:string">
        <xs:enumeration value="BOXCENTER" />
        <xs:enumeration value="OPTCENTER" />
    </xs:restriction>
</xs:simpleType>
<xs:complexType name="TimeStamps">
    <xs:sequence>
        <xs:element name="dateTime">
            <xs:complexType>
                <xs:simpleContent>
                    <xs:extension base="xs:dateTime">
                        <xs:attribute name="step" type="
MsgSteps" use="required" />
                    </xs:extension>
                </xs:simpleContent>
            </xs:complexType>
        </xs:element>
    </xs:sequence>
</xs:complexType>
<xs:complexType name="CurrencyValue">
    <xs:sequence>
        <xs:element name="Value" form="qualified">
            <xs:complexType>
                <xs:simpleContent>
                    <xs:extension base="xs:float">
                        <xs:attribute name="currency" type="
Currencies" use="required" />
                    </xs:extension>
                </xs:simpleContent>
            </xs:complexType>
        </xs:element>
    </xs:sequence>
</xs:complexType>
<xs:complexType name="Software">
    <xs:sequence>
        <xs:element name="name" type="xs:string" />
        <xs:element name="version" type="xs:string" />
    </xs:sequence>
    <xs:attribute name="typeOf" type="SoftwareTypes" use="required" />
</xs:complexType>
<xs:complexType name="ProductCatalog">
    <xs:sequence>
        <xs:element name="name" type="xs:string">
            <xs:annotation>
                <xs:documentation>i.e. "IPRO STANDARD": currently
ignored</xs:documentation>
            </xs:annotation>
        </xs:element>
        <xs:element name="version" type="xs:string">
            <xs:annotation>
                <xs:documentation>i.e. "6.2": currently ignored</
msg:documentation>
            </xs:annotation>
        </xs:element>
        <xs:element name="release" type="xs:string">
            <xs:annotation>
                <xs:documentation>when was the catalogue released or
a release-version: currently ignored</xs:documentation>
            </xs:annotation>
        </xs:element>
    </xs:sequence>
</xs:complexType>
<xs:complexType name="Account">

```

```

        <xs:sequence>
          <xs:element name="id">
            <xs:complexType>
              <xs:simpleContent>
                <xs:extension base="xs:string">
                  <xs:attribute name="memberShipID" type
="xs:long" use="optional" default="1">
                    <xs:annotation>
                      <xs:documentation>CZ:
For further use: Specify the membership-id to support different customerid's for different
brands.</xs:documentation>
                    </xs:annotation>
                  </xs:attribute>
                </xs:extension>
              </xs:simpleContent>
            </xs:complexType>
          </xs:element>
          <xs:element name="name" type="xs:string" minOccurs="0">
            <xs:annotation>
              <xs:documentation>CZ: currently ignored</
xs:documentation>
            </xs:annotation>
          </xs:element>
          <xs:element name="address" type="Address" minOccurs="0">
            <xs:annotation>
              <xs:documentation>CZ: currently ignored</
xs:documentation>
            </xs:annotation>
          </xs:element>
          <xs:element name="contact" type="ContactInfo" minOccurs="0">
            <xs:annotation>
              <xs:documentation>CZ: currently ignored</
xs:documentation>
            </xs:annotation>
          </xs:element>
        </xs:sequence>
      </xs:complexType>
      <xs:complexType name="Address">
        <xs:sequence>
          <xs:element name="addressLine" type="xs:string" minOccurs="0"
maxOccurs="3"/>
          <xs:element name="poBox" type="xs:string" minOccurs="0"/>
          <xs:choice minOccurs="0">
            <xs:element name="province" type="xs:string"/>
            <xs:element name="region" type="xs:string"/>
            <xs:element name="state" type="xs:string"/>
          </xs:choice>
          <xs:element name="city" type="xs:string"/>
          <xs:element name="zip" type="xs:string"/>
          <xs:element name="countryCode" type="xs:string"/>
        </xs:sequence>
      </xs:complexType>
      <xs:complexType name="ContactInfo">
        <xs:sequence>
          <xs:element name="title" type="xs:string" minOccurs="0"/>
          <xs:element name="firstName" type="xs:string" minOccurs="0"/>
          <xs:element name="lastName" type="xs:string"/>
          <xs:element name="phone" type="xs:string" minOccurs="0" maxOccurs="
unbounded"/>
          <xs:element name="fax" type="xs:string" minOccurs="0" maxOccurs="
unbounded"/>
          <xs:element name="eMail" type="xs:string" minOccurs="0" maxOccurs="
unbounded"/>
        </xs:sequence>
      </xs:complexType>
      <xs:complexType name="Lens">
        <xs:sequence>
          <xs:element name="commercialCode" type="xs:string"/>
          <xs:choice>
            <xs:sequence>
              <xs:element name="rxData">
                <xs:complexType>
                  <xs:complexContent>
                    <xs:extension base="RXDataType
"/>

```

```

        </xs:complexContent>
      </xs:complexType>
    </xs:element>
    <xs:element name="coating" minOccurs="0" maxOccurs="
unbounded">
      <xs:complexType>
        <xs:sequence>
          <xs:element name="
commercialCode" type="xs:string"/>
          <xs:element name="minIntensity"
            type="xs:integer" minOccurs="0">
              <xs:annotation>
                <
xs:documentation>Minimum color intensity for tinting as a percentage: currently ignored</
xs:documentation>
              </xs:annotation>
            </xs:element>
          <xs:element name="maxIntensity"
            type="xs:integer" minOccurs="0">
              <xs:annotation>
                <
xs:documentation>Maximum color intensity for tinting as a percentage: currently ignored</
xs:documentation>
              </xs:annotation>
            </xs:element>
        </xs:sequence>
        <xs:attribute name="coatingType" type
="CoatingTypes" use="optional"/>
      </xs:complexType>
    </xs:element>
    <xs:element name="centration" type="Centration"
minOccurs="0"/>
      <xs:element name="geometry" type="GeometryType"/>
      <xs:element name="options" minOccurs="0">
        <xs:complexType>
          <xs:sequence>
            <xs:choice minOccurs="0">
              <xs:element name="
occlusion" type="xs:boolean"/>
              <xs:element name="
frosted" type="xs:boolean">
                <xs:annotation>
                  <
xs:documentation>mattieren</xs:documentation>
                </xs:annotation>
            </xs:choice>
            <xs:element name="slabOff"
minOccurs="0">
              <xs:complexType>
                <
xs:simpleContent>
                  <xs:extension base="xs:boolean">
                    <xs:attribute name="value" use="optional">
                      <xs:simpleType>
                        <xs:restriction base="xs:float">
                          <xs:minInclusive value="1.3"/>
                        </xs:restriction>
                      </xs:simpleType>
                    </xs:attribute>
                  </xs:extension>
                </xs:simpleContent>
              </xs:complexType>
            </xs:element>
          </xs:sequence>
        </xs:complexType>
      </xs:element>
    </xs:complexType>
  </xs:element>

```

```

type="xs:boolean" minOccurs="0">
    <xs:documentation>CZ: currently ignored</xs:documentation>
minOccurs="0">
    <xs:documentation>CZ: currently ignored</xs:documentation>
xs:simpleContent>
xs:extension base="xs:boolean">
    <xs:attribute name="value" type="xs:float" use="optional"/>
xs:extension>
xs:simpleContent>
default="back" minOccurs="0">
    <xs:documentation>CZ: currently ignored</xs:documentation>
base="xs:string">
xs:enumeration value="front"/>
xs:enumeration value="back"/>
xs:restriction>
    <xs:documentation>CZ: Size correction
    for the length of progression zone.</xs:documentation>
    <xs:documentation>CZ: Individual
    gravure text, max five characters. Note: Not all characters supported!</xs:documentation>
    <xs:documentation>for repeated orders:
    currently ignored</xs:documentation>
    <xs:documentation>remakeType
    originalOrderId
    reason</xs:documentation>
    <xs:documentation>experimental</xs:documentation>
maxOccurs="unbounded"/>

```

```

<xs:element name="lenticular"
    <xs:annotation>
    </xs:annotation>
</xs:element>
<xs:element name="aniseicony"
    <xs:annotation>
    </xs:annotation>
    <xs:complexType>
    </xs:complexType>
</xs:element>
<xs:element name="toric"
    <xs:annotation>
    </xs:annotation>
    <xs:simpleType>
    <xs:restriction
    </xs:restriction>
    </xs:simpleType>
</xs:element>
</xs:sequence>
</xs:complexType>
</xs:element>
<xs:element name="frameFit" minOccurs="0">
    <xs:annotation>
    <xs:documentation>CZ: Size correction
    for the length of progression zone.</xs:documentation>
    </xs:annotation>
</xs:element>
<xs:element name="engraving" minOccurs="0">
    <xs:annotation>
    <xs:documentation>CZ: Individual
    gravure text, max five characters. Note: Not all characters supported!</xs:documentation>
    </xs:annotation>
    <xs:simpleType>
    <xs:restriction base="xs:string"/>
    </xs:simpleType>
</xs:element>
</xs:sequence>
<xs:sequence>
    <xs:annotation>
    <xs:documentation>for repeated orders:
    currently ignored</xs:documentation>
    </xs:annotation>
    <xs:element name="remakeType" type="xs:string"/>
    <xs:element name="originalOrderId" type="xs:string"/>
    <xs:element name="reason" type="xs:string"/>
</xs:sequence>
</xs:choice>
<xs:element name="experimental" type="xs:anyType" minOccurs="0"
maxOccurs="unbounded"/>
</xs:sequence>

```

```

<xs:attribute name="side" type="Sides" use="required"/>
<xs:attribute name="balancingLens" type="xs:boolean" use="optional" default
="0"/>
<xs:attribute name="virtualLens" type="xs:boolean" use="optional" default="
0">
    <xs:annotation>
        <xs:documentation>CZ: currently ignored</xs:documentation>
    </xs:annotation>
</xs:attribute>
</xs:complexType>
<xs:complexType name="StockLens">
    <xs:sequence>
        <xs:element name="commercialCode" type="xs:string"/>
        <xs:element name="rxData" type="RXDataType4SL"/>
        <xs:element name="coating" minOccurs="0" maxOccurs="unbounded">
            <xs:complexType>
                <xs:sequence>
                    <xs:element name="commercialCode"/>
                    <xs:element name="minIntensity" type="
xs:integer" minOccurs="0">
                        <xs:annotation>
                            <xs:documentation>Minimum
color intensity for tinting as a percentage</xs:documentation>
                        </xs:annotation>
                    </xs:element>
                    <xs:element name="maxIntensity" type="
xs:integer" minOccurs="0">
                        <xs:annotation>
                            <xs:documentation>Maximum
color intensity for tinting as a percentage</xs:documentation>
                        </xs:annotation>
                    </xs:element>
                </xs:sequence>
                <xs:attribute name="coatingType" type="CoatingTypes"
use="optional"/>
            </xs:complexType>
        </xs:element>
        <xs:element name="diameter">
            <xs:complexType>
                <xs:sequence>
                    <xs:element name="physical">
                        <xs:simpleType>
                            <xs:restriction base="
xs:integer">
                                <xs:minInclusive value
="1"/>
                                <xs:maxInclusive value
="99"/>
                            </xs:restriction>
                        </xs:simpleType>
                    </xs:element>
                    <xs:element name="optical" minOccurs="0">
                        <xs:annotation>
                            <xs:documentation>only if
predecentrated</xs:documentation>
                        </xs:annotation>
                        <xs:simpleType>
                            <xs:restriction base="
xs:integer">
                                <xs:minInclusive value
="1"/>
                                <xs:maxInclusive value
="99"/>
                            </xs:restriction>
                        </xs:simpleType>
                    </xs:element>
                </xs:sequence>
            </xs:complexType>
        </xs:element>
    </xs:sequence>
</xs:complexType>
<xs:complexType name="RXDataType">
    <xs:choice>
        <xs:sequence>
            <xs:element name="sphere">

```

```

        <xs:simpleType>
          <xs:restriction base="xs:float">
            <xs:minInclusive value="-50"/>
            <xs:maxInclusive value="50"/>
          </xs:restriction>
        </xs:simpleType>
      </xs:element>
    </xs:element>
    <xs:element name="cylinder" type="Cylinder" minOccurs="0"/>
    <xs:element name="addition" minOccurs="0">
      <xs:simpleType>
        <xs:restriction base="xs:float">
          <xs:minInclusive value="0.25"/>
        </xs:restriction>
      </xs:simpleType>
    </xs:element>
    <xs:element name="prism" minOccurs="0" maxOccurs="2">
      <xs:complexType>
        <xs:complexContent>
          <xs:extension base="Prism">
            <xs:attribute name="
pupillary-distance-correction" use="optional">
          </xs:extension>
        </xs:complexContent>
      </xs:complexType>
    </xs:element>
  </xs:sequence>
  <xs:element name="trialFrame" type="TrialFrame">
    <xs:annotation>
      <xs:documentation>centration data needed: currently
ignored</xs:documentation>
    </xs:annotation>
  </xs:element>
</xs:choice>
</xs:complexType>
<xs:complexType name="RXDataType4SL">
  <xs:sequence>
    <xs:element name="sphere">
      <xs:simpleType>
        <xs:restriction base="xs:float">
          <xs:minInclusive value="-50"/>
          <xs:maxInclusive value="50"/>
        </xs:restriction>
      </xs:simpleType>
    </xs:element>
    <xs:element name="cylinder" type="Cylinder" minOccurs="0"/>
    <xs:element name="addition" minOccurs="0">
      <xs:simpleType>
        <xs:restriction base="xs:float">
          <xs:minInclusive value="0.25"/>
        </xs:restriction>
      </xs:simpleType>
    </xs:element>
  </xs:sequence>
</xs:complexType>
<xs:complexType name="Centration">
  <xs:sequence>
    <xs:element name="monocularCentrationDistance">
      <xs:annotation>
        <xs:documentation>CZ: see @reference annotation</
xs:documentation>
      </xs:annotation>
    </xs:element>
  </xs:sequence>
</xs:complexType>

```

```

        <xs:simpleContent>
          <xs:extension base="xs:float">
            <xs:attribute name="reference" type="
OCReferences" use="required">
              <xs:annotation>
                <xs:documentation>CZ:
Currently only FAR supported, NEAR will be ignored!</xs:documentation>
              </xs:annotation>
            </xs:attribute>
          </xs:extension>
        </xs:simpleContent>
      </xs:complexType>
    </xs:element>
    <xs:element name="height">
      <xs:annotation>
        <xs:documentation>CZ: see @reference annotation</
xs:documentation>
      </xs:annotation>
    </xs:complexType>
    <xs:simpleContent>
      <xs:extension base="xs:float">
        <xs:attribute name="reference" type="
OCReferences" use="required">
          <xs:annotation>
            <xs:documentation>FAR -
> Y-> H</xs:documentation>
          </xs:annotation>
        </xs:attribute>
      <xs:attribute name="referenceHeight"
type="HeightReferences" use="required"/>
    </xs:extension>
  </xs:simpleContent>
</xs:complexType>
</xs:element>
<xs:element name="backVertexDistance" minOccurs="0">
  <xs:complexType>
    <xs:simpleContent>
      <xs:extension base="xs:float"/>
    </xs:simpleContent>
  </xs:complexType>
</xs:element>
</xs:sequence>
</xs:complexType>
<xs:complexType name="Shape">
  <xs:sequence>
    <xs:element name="perimeter" type="xs:float">
      <xs:annotation>
        <xs:documentation>CZ: currently ignored</
xs:documentation>
      </xs:annotation>
    </xs:element>
    <xs:element name="points">
      <xs:complexType>
        <xs:choice>
          <xs:element name="pPoints" type="PPoint"
minOccurs="18" maxOccurs="unbounded">
            <xs:annotation>
              <xs:documentation>CZ: In an
order (header[@msgType=ORDER]) any number of polar coordinates other than 36 will result
in an exception thrown to the client. YOU HAVE BEEN WARNED!</xs:documentation>
            </xs:annotation>
          </xs:element>
          <xs:element name="cPoints" type="CPoint"
minOccurs="18" maxOccurs="unbounded">
            <xs:annotation>
              <xs:documentation>CZ:
currently ignored - use polar coordinates instead!</xs:documentation>
            </xs:annotation>
          </xs:element>
        </xs:choice>
      </xs:complexType>
    </xs:element>
    <xs:attribute name="dimensions" type="xs:integer" use
="required"/>
    <xs:attribute name="reference" type="References" use
="optional"/>
  </xs:complexType>

```

```

        </xs:element>
      </xs:sequence>
    </xs:complexType>
  <xs:complexType name="CPoint">
    <xs:sequence>
      <xs:element name="x" type="xs:float">
        <xs:annotation>
          <xs:documentation>mm</xs:documentation>
        </xs:annotation>
      </xs:element>
      <xs:element name="y" type="xs:float">
        <xs:annotation>
          <xs:documentation>mm</xs:documentation>
        </xs:annotation>
      </xs:element>
      <xs:element name="z" type="xs:float" minOccurs="0">
        <xs:annotation>
          <xs:documentation>mm</xs:documentation>
        </xs:annotation>
      </xs:element>
    </xs:sequence>
  </xs:complexType>
  <xs:complexType name="PPoint">
    <xs:sequence>
      <xs:element name="angle">
        <xs:complexType>
          <xs:simpleContent>
            <xs:extension base="xs:float">
              <xs:attribute name="dimension" type="
AngleDimension" use="optional" default="DEG"/>
            </xs:extension>
          </xs:simpleContent>
        </xs:complexType>
      </xs:element>
      <xs:element name="radius" type="xs:float">
        <xs:annotation>
          <xs:documentation>mm</xs:documentation>
        </xs:annotation>
      </xs:element>
      <xs:element name="z" type="xs:float" minOccurs="0">
        <xs:annotation>
          <xs:documentation>mm: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
    </xs:sequence>
  </xs:complexType>
  <xs:complexType name="Size">
    <xs:sequence>
      <xs:element name="width" type="xs:float"/>
      <xs:element name="height" type="xs:float"/>
      <xs:element name="bridge" type="xs:float"/>
      <xs:element name="templeLength" type="xs:float" minOccurs="0"/>
    </xs:sequence>
  </xs:complexType>
  <xs:complexType name="Bevel">
    <xs:sequence>
      <xs:element name="type" type="BevelTypes"/>
      <xs:element name="position" minOccurs="0" maxOccurs="2">
        <xs:annotation>
          <xs:documentation>in case of FRONT, BACK: [mm]
distance to front/back. In case of RELATED: [%] front/back-dist.
</xs:documentation>
        </xs:annotation>
      </xs:element>
    </xs:sequence>
  </xs:complexType>
  <xs:extension base="xs:float">
    <xs:attribute name="posType" type="
BevelPosType" use="optional" default="AUTO"/>
    <xs:attribute name="side" type="Sides"
use="optional"/>
  </xs:extension>
</xs:simpleContent>
</xs:complexType>
</xs:element>

```

```
<xs:element name="grooveDepth" type="xs:float" minOccurs="0">
  <xs:annotation>
    <xs:documentation>CZ: currently ignored</
xs:documentation>
    </xs:annotation>
  </xs:element>
  <xs:element name="grooveWidth" type="xs:float" minOccurs="0">
    <xs:annotation>
      <xs:documentation>CZ: currently ignored</
xs:documentation>
      </xs:annotation>
    </xs:element>
  </xs:sequence>
</xs:complexType>
<xs:complexType name="Curve">
  <xs:choice>
    <xs:element name="flat" type="xs:boolean"/>
    <xs:element name="moreCurved" type="xs:boolean">
      <xs:annotation>
        <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
    <xs:element name="baseCurve" type="xs:float">
      <xs:annotation>
        <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
    <xs:element name="planoConcave" type="xs:boolean">
      <xs:annotation>
        <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
    <xs:element name="planoConvex" type="xs:boolean">
      <xs:annotation>
        <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
    <xs:element name="biConcave" type="xs:boolean">
      <xs:annotation>
        <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
    <xs:element name="biConvex" type="xs:boolean">
      <xs:annotation>
        <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
  </xs:choice>
</xs:complexType>
<xs:complexType name="Header">
  <xs:sequence>
    <xs:element name="cultureId" type="xs:string">
      <xs:annotation>
        <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
    <xs:element name="customersOrderId" type="xs:string">
      <xs:annotation>
        <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
    <xs:element name="distributorsOrderId" type="xs:string">
      <xs:annotation>
        <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
  </xs:sequence>
</xs:complexType>
```

```

        <xs:element name="timeStamps" type="TimeStamps" maxOccurs="unbounded"
">
            <xs:annotation>
                <xs:documentation>CZ: currently ignored</
xs:documentation>
            </xs:annotation>
        </xs:element>
        <xs:element name="remark" type="xs:string" minOccurs="0">
            <xs:annotation>
                <xs:documentation>CZ: Any comments or remarks are
effectively ignored!</xs:documentation>
            </xs:annotation>
        </xs:element>
        <xs:element name="orderParties" maxOccurs="unbounded">
            <xs:annotation>
                <xs:documentation>CZ: at least one entry with @role
ORIGINATOR or INVOICETO should be present, unless present under items/parties; serves as
default - if nothing is specified in items branch, these entries are used</
xs:documentation>
            </xs:annotation>
            <xs:complexType>
                <xs:complexContent>
                    <xs:extension base="Account">
                        <xs:attribute name="role" type="Roles"
use="required"/>
                    </xs:extension>
                </xs:complexContent>
            </xs:complexType>
            <xs:element name="software" type="Software" maxOccurs="unbounded"/>
            <xs:element name="productCatalog" type="ProductCatalog"/>
            <xs:element name="portalOrderId" type="xs:string">
                <xs:annotation>
                    <xs:documentation>retailer's order id: currently
ignored</xs:documentation>
                </xs:annotation>
            </xs:element>
        </xs:sequence>
        <xs:attribute name="msgType" type="MsgTypes" use="optional" default="
REQUEST"/>
        <xs:attribute name="msgState" type="MsgStates" use="optional" default="NEW
"/>
        <xs:attribute name="testIndicator" type="xs:boolean" use="optional" default
="0"/>
    </xs:complexType>
    <xs:complexType name="Item">
        <xs:sequence>
            <xs:element name="remark" type="xs:string" minOccurs="0">
                <xs:annotation>
                    <xs:documentation>CZ: Any comments or remarks are
effectively ignored!</xs:documentation>
                </xs:annotation>
            </xs:element>
            <xs:element name="parties" minOccurs="0" maxOccurs="unbounded">
                <xs:annotation>
                    <xs:documentation>if not given use defaults from
header: selection rules:
@role ORIGINATOR will receive invoice if no @role INVOICETO is presentno @role SHIPTO is
present, the recipient of the invoice will also become recipient of the goods</
xs:documentation>
                </xs:annotation>
            </xs:complexType>
                <xs:complexContent>
                    <xs:extension base="Account">
                        <xs:attribute name="role" type="Roles"
use="required"/>
                    </xs:extension>
                </xs:complexContent>
            </xs:complexType>
            <xs:element name="referenceNo" type="xs:string">
                <xs:annotation>
                    <xs:documentation>i.e. buyer's job no.</
xs:documentation>
                </xs:annotation>
            </xs:element>
        </xs:sequence>
    </xs:complexType>

```

```

</xs:element>
<xs:element name="referenceText" type="xs:string" minOccurs="0">
  <xs:annotation>
    <xs:documentation>i.e. buyer's commission: currently
ignored - use referenceNo also for textual references (commission)</xs:documentation>
  </xs:annotation>
</xs:element>
<xs:element name="requestedDate" type="xs:date" minOccurs="0">
  <xs:annotation>
    <xs:documentation>CZ: currently ignored</
xs:documentation>
  </xs:annotation>
</xs:element>
<xs:element name="deliveryMethod">
  <xs:annotation>
    <xs:documentation>CZ: currently ignored</
xs:documentation>
  </xs:annotation>
<xs:complexType>
  <xs:simpleContent>
    <xs:extension base="xs:string">
      <xs:attribute name="deliveryType" type
="DeliveryMethods" use="optional">
        <xs:annotation>
          <xs:documentation>
enumeration of predefined methodes
</xs:documentation>
        </xs:annotation>
      </xs:attribute>
    </xs:extension>
  </xs:simpleContent>
</xs:complexType>
</xs:element>
<xs:element name="manufacturer" type="xs:string">
  <xs:annotation>
    <xs:documentation>CZ: currently ignored</
xs:documentation>
  </xs:annotation>
</xs:element>
<xs:choice>
  <xs:element name="pair" type="Pair"/>
  <xs:element name="stockLens">
    <xs:annotation>
      <xs:documentation>CZ: currently ignored</
xs:documentation>
    </xs:annotation>
  </xs:element>
  <xs:complexType>
    <xs:complexContent>
      <xs:extension base="StockLens">
        <xs:attribute name="quantity"
type="xs:integer" use="required"/>
        <xs:attribute name="side" type
="Sides" use="required"/>
      </xs:extension>
    </xs:complexContent>
  </xs:complexType>
</xs:element>
<xs:element name="semi">
  <xs:annotation>
    <xs:documentation>CZ: currently ignored</
xs:documentation>
  </xs:annotation>
</xs:element>
</xs:choice>
<xs:element name="options" minOccurs="0">
  <xs:annotation>
    <xs:documentation>CZ: currently ignored</
xs:documentation>
  </xs:annotation>
<xs:complexType>
  <xs:sequence>
    <xs:element name="insurance" type="xs:integer
" minOccurs="0"/>
  </xs:sequence>
</xs:complexType>

```

```

        </xs:element>
      </xs:sequence>
    </xs:complexType>
  <xs:complexType name="Frame">
    <xs:sequence>
      <xs:element name="material" type="FrameMaterials">
        <xs:annotation>
          <xs:documentation>CZ: currently recognized: METAL,
PLASTIC, NYLOR, SPECIAL</xs:documentation>
        </xs:annotation>
      </xs:element>
      <xs:element name="manufacturer" type="xs:string" minOccurs="0"/>
      <xs:element name="brand" type="xs:string" minOccurs="0">
        <xs:annotation>
          <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
      <xs:element name="model" type="xs:string" minOccurs="0"/>
      <xs:element name="size" type="Size" minOccurs="0">
        <xs:annotation>
          <xs:documentation>Nominal size information as given
from frame manufacturer: currently ignored - use size elements under shape!</
xs:documentation>
        </xs:annotation>
      </xs:element>
      <xs:element name="color" type="xs:string" minOccurs="0">
        <xs:annotation>
          <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
      <xs:element name="commercialCode" type="xs:string" minOccurs="0">
        <xs:annotation>
          <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
      </xs:element>
      <xs:element name="shape" minOccurs="0">
        <xs:complexType>
          <xs:sequence>
            <xs:choice>
              <xs:element name="catalog">
                <xs:annotation>
                  <xs:documentation>for
thicknessreduction: currently ignored</xs:documentation>
                </xs:annotation>
              </xs:element>
              <xs:sequence>
                <xs:element
name="shapeId" type="xs:string"/>
              </xs:sequence>
            </xs:choice>
          </xs:complexType>
        </xs:element>
        <xs:element name="explicit" type="
Shape" maxOccurs="2">
          <xs:annotation>
            <xs:documentation>CZ:
If explicit occurs once, data is assumed to belong to the right lens - no matter if there
is only a right lens, only a left lens or two lenses in the document. If explicit occurs
twice, the first occurrence is assumed to belong to the right lens, the second to the left
lens respectively - again no matter how many lenses are really contained in the document
under pair/lens</xs:documentation>
          </xs:annotation>
        </xs:element>
        <xs:element name="tracerData">
          <xs:annotation>
            <xs:documentation>
necessary for remoteEdging/Presize</xs:documentation>
          </xs:annotation>
        </xs:complexType>
      </xs:sequence>
    </xs:element>
    name="tracerBrand" type="xs:string">

```

<

```
xs:annotation>
<xs:documentation>i.e. "Briot", "Essilor", "Nidek"</xs:documentation>
</
xs:annotation>
</xs:element>
<xs:element
name="tracerType">
<
xs:annotation>
</
<xs:documentation>i.e. "Scanform Net 2", "Phi": currently ignored</xs:documentation>
</
xs:annotation>
<
xs:simpleType>
<xs:restriction base="xs:string">
  <xs:enumeration value="Hoya 3DFT"/>
  <xs:enumeration value="Hoya GT1000"/>
  <xs:enumeration value="Hoya GT3000"/>
  <xs:enumeration value="Hoya GT3000 OMA"/>
  <xs:enumeration value="Hoya UT1000"/>
  <xs:enumeration value="Essilor Gamma"/>
  <xs:enumeration value="Essilor PHI PCCOM"/>
  <xs:enumeration value="Essilor PHI OMA"/>
  <xs:enumeration value="Essilor Kappa PCCOM"/>
  <xs:enumeration value="Essilor Kappa OMA"/>
  <xs:enumeration value="National Optronics 4T"/>
  <xs:enumeration value="Briot ScanfomNet I"/>
  <xs:enumeration value="Briot ScanfomNet II"/>
  <xs:enumeration value="Weco 3DFT+"/>
  <xs:enumeration value="Weco Trace II"/>
  <xs:enumeration value="Weco Trace III"/>
  <xs:enumeration value="Nidek LT700"/>
  <xs:enumeration value="Nidek LT900"/>
  <xs:enumeration value="Nidek LT1000"/>
  <xs:enumeration value="Indo NDK"/>
  <xs:enumeration value="Indo OMA"/>
  <xs:enumeration value="Topcon FR50"/>
  <xs:enumeration value="Takubomatic FD80"/>
  <xs:enumeration value="Unknown"/>
</xs:restriction>
</
xs:simpleType>
</xs:element>
<xs:element
name="tracerVersion" type="xs:string">
<
xs:annotation>
```

```

<xs:documentation>CZ: currently ignored</xs:documentation>
xs:annotation>
</xs:element>
name="tracerID" type="xs:string" minOccurs="0">
<xs:element
xs:annotation>
</xs:element>
<xs:documentation>CZ: Serial number of the tracer</xs:documentation>
xs:annotation>
</xs:element>
name="binaries">
<xs:element
xs:complexType>
<xs:simpleContent>
  <xs:extension base="xs:hexBinary">
    <xs:attribute name="format" type="xs:string">
      <xs:annotation>
        <xs:documentation>use common values. i.e. "OMA", "NIDEK",
"MO1", "WECO"...</xs:documentation>
      </xs:annotation>
    </xs:attribute>
  </xs:extension>
</xs:simpleContent>
</xs:complexType>
</xs:element>
</xs:sequence>
</xs:complexType>
</xs:element>
</xs:choice>
<xs:element name="adjustion" type="xs:float"
minOccurs="0">
  <xs:annotation>
    <xs:documentation>general size
adjustion parameter: Size correction in the Nidek data set.</xs:documentation>
  </xs:annotation>
</xs:element>
</xs:sequence>
</xs:complexType>
</xs:element>
<xs:element name="boxWidth" type="xs:float" minOccurs="0">
  <xs:annotation>
    <xs:documentation>measured box width for size
transformations</xs:documentation>
  </xs:annotation>
</xs:element>
<xs:element name="boxHeight" type="xs:float" minOccurs="0">
  <xs:annotation>
    <xs:documentation>measured box height for size
transformations</xs:documentation>
  </xs:annotation>
</xs:element>
<xs:element name="distanceBetweenLenses" type="xs:float" minOccurs="
0">
  <xs:annotation>
    <xs:documentation>measured dbl for calculation
purposes</xs:documentation>
  </xs:annotation>
</xs:element>
<xs:element name="drillHoles" minOccurs="0" maxOccurs="2">
  <xs:annotation>

```

```

        <xs:documentation>CZ: If attribute side is UNDEFINED
and drillHoles occurs only once the drillhole values are mirrored from right to left side
</xs:documentation>
        </xs:annotation>
        <xs:complexType>
            <xs:complexContent>
                <xs:extension base="DrillHoles">
                    <xs:attribute name="side" type="Sides"
use="prohibited"/>
                </xs:extension>
            </xs:complexContent>
        </xs:complexType>
    </xs:element>
    <xs:element name="pantoscopicAngle" minOccurs="0">
        <xs:complexType>
            <xs:simpleContent>
                <xs:extension base="xs:float">
                    <xs:attribute name="dimension" type="
AngleDimension" use="optional" default="DEG"/>
                </xs:extension>
            </xs:simpleContent>
        </xs:complexType>
    </xs:element>
    <xs:element name="frameBowAngle" minOccurs="0">
        <xs:annotation>
            <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
        <xs:complexType>
            <xs:simpleContent>
                <xs:extension base="xs:float">
                    <xs:attribute name="dimension" type="
AngleDimension" use="optional" default="DEG"/>
                </xs:extension>
            </xs:simpleContent>
        </xs:complexType>
    </xs:element>
    <xs:element name="remark" type="xs:string" minOccurs="0">
        <xs:annotation>
            <xs:documentation>CZ: Any comments or remarks are
effectively ignored!</xs:documentation>
        </xs:annotation>
    </xs:element>
    <xs:element name="frameCurve" type="xs:float" minOccurs="0">
        <xs:annotation>
            <xs:documentation>CZ: currently ignored</
xs:documentation>
        </xs:annotation>
    </xs:element>
    </xs:sequence>
    <xs:attribute name="quantity" type="xs:integer" use="required"/>
</xs:complexType>
<xs:complexType name="TrialFrame">
    <xs:sequence>
        <xs:element name="sphere" minOccurs="0">
            <xs:simpleType>
                <xs:restriction base="xs:float">
                    <xs:minInclusive value="-50"/>
                    <xs:maxInclusive value="50"/>
                </xs:restriction>
            </xs:simpleType>
        </xs:element>
        <xs:element name="cylinder" type="Cylinder" minOccurs="0"/>
        <xs:element name="prism" type="Prism" maxOccurs="2"/>
        <xs:element name="decentration">
            <xs:complexType>
                <xs:sequence>
                    <xs:element name="horizontal" type="xs:float"
"/>
                    <xs:element name="vertical" type="xs:float"/>
                </xs:sequence>
            </xs:complexType>
        </xs:element>
        <xs:element name="back-vertex-distance" type="xs:float"/>
    </xs:sequence>

```

```

</xs:complexType>
<xs:complexType name="Prism">
  <xs:sequence>
    <xs:element name="power" type="xs:float" />
    <xs:element name="base">
      <xs:simpleType>
        <xs:restriction base="xs:float">
          <xs:minInclusive value="0.0" />
          <xs:maxInclusive value="360.0" />
        </xs:restriction>
      </xs:simpleType>
    </xs:element>
  </xs:sequence>
</xs:complexType>
<xs:complexType name="Cylinder">
  <xs:sequence>
    <xs:element name="power" type="xs:float" />
    <xs:element name="base">
      <xs:simpleType>
        <xs:restriction base="xs:float">
          <xs:minInclusive value="0.0" />
          <xs:maxInclusive value="360.0" />
        </xs:restriction>
      </xs:simpleType>
    </xs:element>
  </xs:sequence>
</xs:complexType>
<xs:complexType name="DrillHoles">
  <xs:choice>
    <xs:element name="pDrillHole" maxOccurs="4">
      <xs:annotation>
        <xs:documentation>CZ: currently ignored - use
cDrillHole instead</xs:documentation>
      </xs:annotation>
      <xs:complexType>
        <xs:sequence>
          <xs:element name="point" maxOccurs="2">
            <xs:complexType>
              <xs:sequence>
                <xs:element name="angle"
" >
                </xs:sequence>
              </xs:complexType>
            </xs:element>
          </xs:sequence>
        </xs:complexType>
      </xs:complexType>
    </xs:choice>
  </xs:complexType>
  <xs:extension base="xs:float" />
  <xs:simpleContent>
    </xs:element>
  </xs:complexType>
  <xs:element name="radius" type="xs:float">
    <xs:annotation>
      <xs:documentation>mm</xs:documentation>
    </xs:annotation>
  </xs:element>
  <xs:element name="z"
type="xs:float" minOccurs="0">
    <xs:annotation>
      <xs:documentation>mm</xs:documentation>
    </xs:annotation>
  </xs:element>
</xs:complexType>
</xs:sequence>
</xs:complexType>
</xs:element>
</xs:sequence>
</xs:complexType>
</xs:element>
</xs:sequence>
</xs:complexType>
</xs:element>
<xs:element name="cDrillHole" maxOccurs="4">

```

```

<xs:complexType>
  <xs:sequence>
    <xs:element name="point" maxOccurs="2">
      <xs:complexType>
        <xs:sequence>
          <xs:element name="x"
            type="xs:float">
              <xs:annotation>
                <
                </xs:annotation>
              </xs:element>
            <xs:element name="y"
              type="xs:float">
                <xs:annotation>
                  <
                  </xs:annotation>
                </xs:element>
              <xs:element name="z"
                type="xs:float" minOccurs="0">
                  <xs:annotation>
                    <
                    </xs:annotation>
                  </xs:element>
                <xs:documentation>mm: currently ignored</xs:documentation>
              </xs:element>
            </xs:sequence>
          </xs:complexType>
        </xs:element>
      </xs:sequence>
    </xs:complexType>
  </xs:choice>
</xs:complexType>
<xs:complexType name="Pair">
  <xs:sequence>
    <xs:element name="patient" minOccurs="0">
      <xs:complexType>
        <xs:complexContent>
          <xs:extension base="Account">
            <xs:sequence minOccurs="0">
              <xs:element name="
classification" minOccurs="0" maxOccurs="unbounded">
                <xs:annotation>
                  <
                  </xs:annotation>
                </xs:complexType>
              </xs:sequence>
            </xs:complexContent>
          </xs:extension>
        </xs:complexType>
      </xs:element>
    <xs:extension base="xs:string">
      <xs:attribute name="typeOfClass" type="xs:string" use="required"/>
    </xs:extension>
  </xs:sequence>
  </xs:complexType>
  <xs:element name="mailAllowed"
    type="xs:boolean" minOccurs="0">
    <xs:annotation>
      <
      </xs:annotation>
    </xs:element>
  <xs:element name="dateOfBirth"
    type="xs:date" minOccurs="0">
    <xs:annotation>
      <
      </xs:annotation>
    </xs:element>
  <xs:documentation>CZ: currently ignored</xs:documentation>
</xs:complexType>

```

```

        </xs:annotation>
    </xs:element>
    <xs:element name="
interpupillaryDistance" type="xs:float" minOccurs="0">
        <xs:annotation>
            <
xs:documentation>CZ: currently ignored</xs:documentation>
        </xs:annotation>
    </xs:element>
    <xs:element name="
nearObjectDistance" type="xs:float" minOccurs="0"/>
        </xs:sequence>
    </xs:extension>
    </xs:complexContent>
    </xs:complexType>
</xs:element>
<xs:element name="lens" maxOccurs="2">
    <xs:complexType>
        <xs:complexContent>
            <xs:extension base="Lens">
                <xs:attribute name="quantity" type="
xs:integer" use="required">
                    <xs:annotation>
                        <xs:documentation>CZ:
any quantity other than 1 will result in an exception thrown to the client! YOU HAVE BEEN
WARNED!</xs:documentation>
                    </xs:annotation>
                </xs:attribute>
            </xs:extension>
        </xs:complexContent>
    </xs:complexType>
</xs:element>
<xs:element name="frame" type="Frame" minOccurs="0"/>
<xs:element name="edging" minOccurs="0">
    <xs:complexType>
        <xs:sequence>
            <xs:element name="bevel">
                <xs:complexType>
                    <xs:complexContent>
                        <xs:extension base="
Bevel"/>
                    </xs:complexContent>
                </xs:complexType>
            </xs:element>
            <xs:element name="polish" type="xs:boolean"/>
            <xs:element name="chamfer" type="xs:integer">
                <xs:annotation>
                    <xs:documentation>Bitmap:
0=no, 1=front, 2=back, 4=strong, : currently ignored</xs:documentation>
                </xs:annotation>
            </xs:element>
            <xs:element name="ignoreFlags" type="
xs:string" minOccurs="0" maxOccurs="unbounded">
                <xs:annotation>
                    <xs:documentation>CZ:
currently ignored</xs:documentation>
                </xs:annotation>
            </xs:element>
        </xs:sequence>
        <xs:attribute name="edgingType" type="EdgingType" use
="required"/>
    </xs:complexType>
</xs:element>
<xs:element name="remark" type="xs:string" minOccurs="0">
    <xs:annotation>
        <xs:documentation>CZ: Any comments or remarks are
effectively ignored!</xs:documentation>
    </xs:annotation>
</xs:element>
</xs:sequence>
<xs:attribute name="thicknessMatching" type="xs:integer" use="optional">
    <xs:annotation>
        <xs:documentation>0=UNDEF, 1=MID, 2=EDGE</xs:documentation>
        <xs:documentation>CZ: value 2 currently ignored</
xs:documentation>
    </xs:documentation>

```

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```

xs:maxInclusive value="40.0"/>
</xs:restriction>
</xs:simpleType>
</xs:element>
<xs:element name="direction">
  <xs:simpleType>
    <xs:restriction base="
xs:float">
      <
xs:minInclusive value="0.0"/>
      <
xs:maxInclusive value="360.0"/>
    </xs:restriction>
  </xs:simpleType>
</xs:element>
</xs:sequence>
<xs:sequence>
  <xs:annotation>
    <xs:documentation>CZ:
currently ignored - use length - direction instead!</xs:documentation>
  </xs:annotation>
  <xs:element name="horizontal" type="
xs:float"/>
  <xs:element name="vertical" type="
xs:float"/>
  </xs:sequence>
</xs:choice>
<xs:attribute name="origin" use="optional" default="
internal">
  <xs:simpleType>
    <xs:restriction base="xs:string">
      <xs:enumeration value="
internal"/>
      <xs:enumeration value="
customer"/>
    </xs:restriction>
  </xs:simpleType>
</xs:attribute>
</xs:complexType>
</xs:element>
<xs:element name="thickness" minOccurs="0">
  <xs:annotation>
    <xs:documentation>this means "minimum thickness". The
real thickness can even be larger.: minimum center thickness</xs:documentation>
  </xs:annotation>
  <xs:complexType>
    <xs:simpleContent>
      <xs:extension base="xs:float">
        <xs:attribute name="reference" type="
ThicknessReferences" use="required"/>
      </xs:extension>
    </xs:simpleContent>
  </xs:complexType>
</xs:element>
<xs:element name="thicknessReduction" type="xs:boolean" minOccurs="0
">
  <xs:annotation>
    <xs:documentation>CZ: Flag for our Optima</
xs:documentation>
  </xs:annotation>
</xs:element>
<xs:element name="curve" type="Curve" minOccurs="0"/>
<xs:element name="inset" type="xs:float" minOccurs="0">
  <xs:annotation>
    <xs:documentation>for lenses with variable insets:
currently ignored</xs:documentation>
  </xs:annotation>
</xs:element>
<xs:element name="upset" type="xs:float" minOccurs="0">
  <xs:annotation>
    <xs:documentation>for lenses with variable upsets:
currently ignored</xs:documentation>
  </xs:annotation>
</xs:element>
<xs:element name="progressionLength" type="xs:float" minOccurs="0">

```

```
        <xs:annotation>
          <xs:documentation>for progressive lenses with
variable length of progressionzone: currently ignored</xs:documentation>
        </xs:annotation>
      </xs:element>
    </xs:sequence>
  </xs:complexType>
</xs:schema>
```

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